

SPARE PARTS CATALOG

RESERVDLSKATALOG

Explorer Mini

Permobil power wheelchair
Permobil elrullstol

permobil

Contents

Chassis.....

Seat

Stickers.....


Produced and published by Permobil AB, Sweden
Release date: 2019-12
PAB: 1300:1

Innehållsförteckning


Chassi.....

Sits

Klistermärken



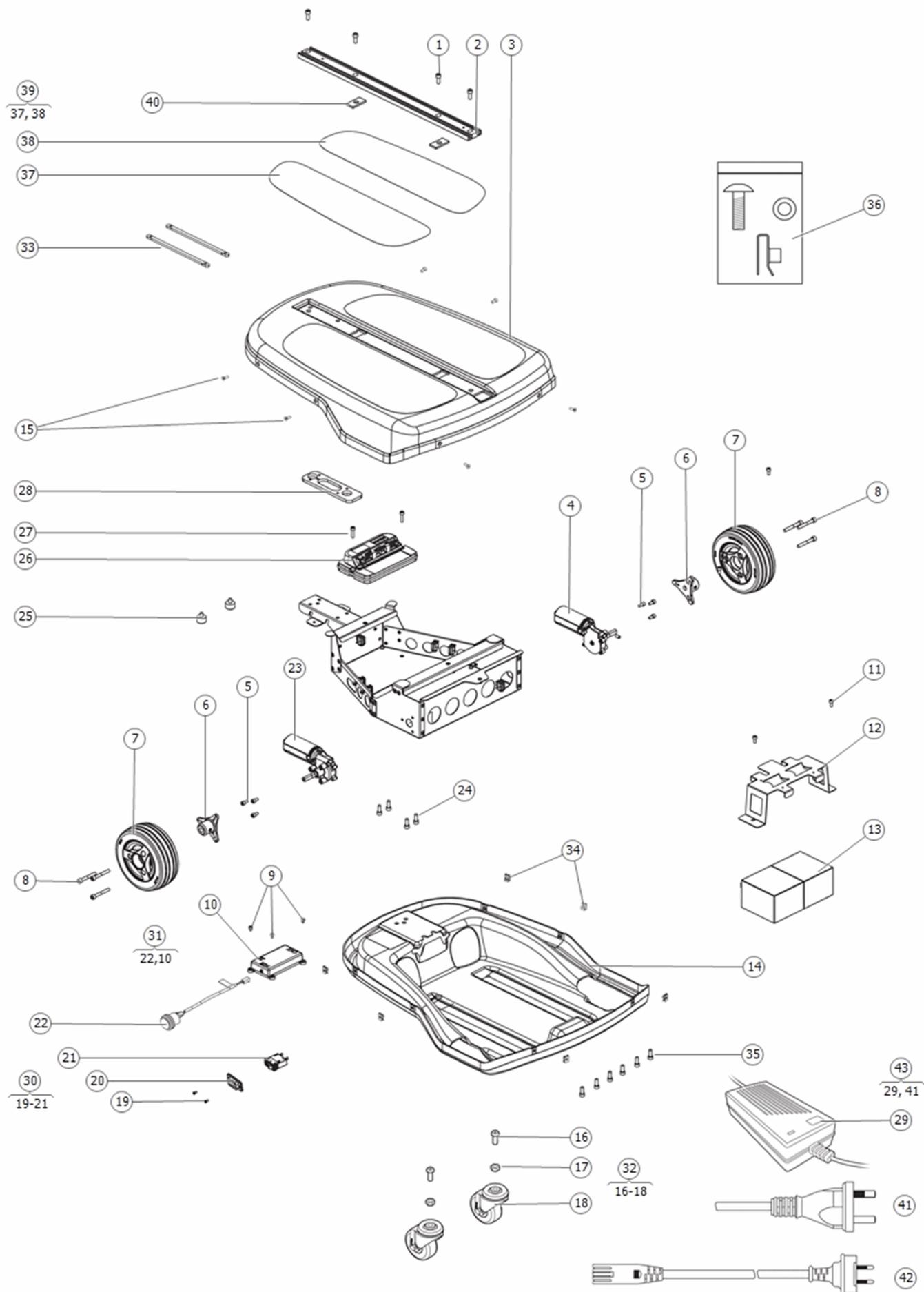


Pos	Qty	Stock Code	Description	Remark
1	4	600743-99-0	Bolt MC6S M6x16 fzb	
2	1	333945	Unitrack	
3	1	1835048	Cover, Explorer Mini	
4	1	1835045	Motor 125rpm 24V, Left	
5	6	600189-99-0	Bolt MC6S 5x10 fzb	
6	1	334167	Hub	
7	1	1836270	Wheel kit 150x50	
8	3	600197-99-0	BOLT MC6S 6x45 fzb *	
9	3	601282-99-0	Hexagon socket head screw, M4x5	
10	1	334404	Electronics box	
11	2	600189-99-0	BOLT MC6S 5x10 fzb *	
12	1	334152	Battery holder	
13	2	1835006	Battery 12V 5Ah	
14	1	1835049	Cover	
15	6	600841-99-0	Hexagon socket countersunk head screw, M4x12	
16	2	601952-99-0	Hexagon socket button head screw, M10x25	
17	2	600229-99-0	LOCK NUT M10 FZB *	
18	2	602185-99-0	WHEEL FOOTPL AR 1101075-0	
19	2	614508	Tapping screw	
20	1	615780	Cover	
21	1	615781	Circuit breaker	
22	1	1836269	Cable, battery saver switch	
23	1	1835044	Motor125rpm 24V, right	
24	4	600743-99-0	Hexagon socket head cap screw, M6x16	
25	2	602639-99-0	RUBBER BLOCK MN60 M6*10 3018055	
26	1	1835007	Power module, nVR2 PM50	
27	2	600543-99-0	BOLT MC6S 5x20 fzb *	
28	1	333876	Plate Explorer Mini	
29	1	1834971	Battery charger ROW	
30	1	1835046	Circuit breaker and switch	
31	1	1835050	Battery saver incl. battery saver button	
32	2	1835047	Pivot wheel kit	
33	2	334491	Long nut	
34	6	615892	Clip nut	
35	6	615779-99-0	Hexagon socket button screw M5x12	
36	1	1835051	Screws, nuts and clips kit for seat and chassis	
37	1	334599	Footpad friction sticker, right	
38	1	334598	Footpad friction sticker, left	
39	1	1836052	Footpad friction kit	
40	2	336290	Spacer	
41	1	614399	Cable for AUS	
42	1	1833454	Charger cable	
43	1	1836380	Battery charger ROW incl. cable for AUS	

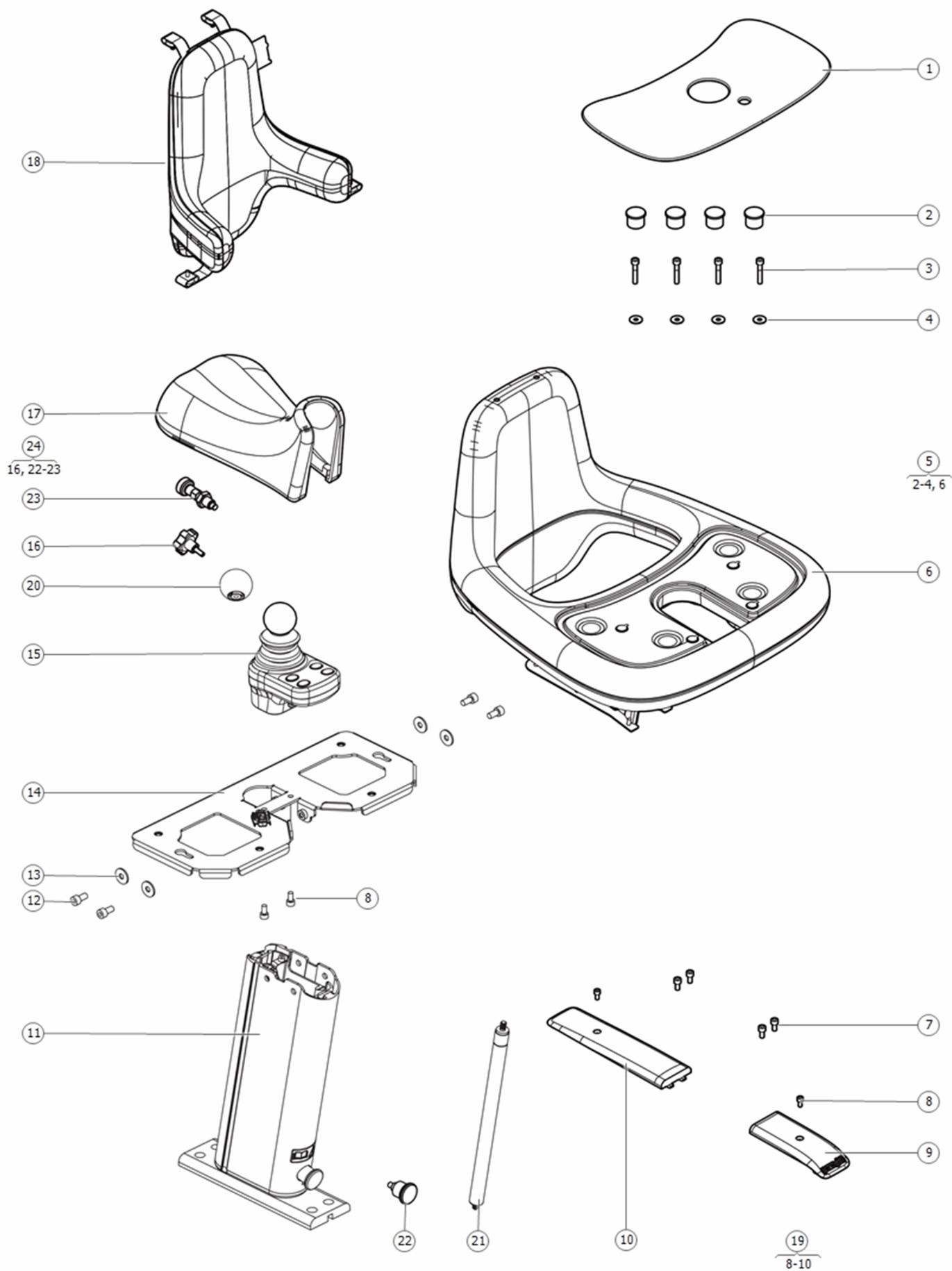

Pos	Qty	Stock Code	Description	Remark
1	1	1835055	Tray (stickers are not included)	
2	4	615884	Plastic plug	
3	4	601333-99-0	Hexagon socket head cap screw, M6x35	
4	4	612675-99-0	Washer, ISO 7093-1 6,4x18x1,6 fzb	
5	1	1835053	Table incl. screws	
6	1	334440	Table	
7	4	600192-99-0	Bolt MC6S 6x12 fzb	
8	2	600189-99-0	Bolt MC6S 5x10 fzb	
9	1	334495	Cover Unitrack, front	
10	1	334462	Cover Unitrack, rear	
11	1	334490	Inner/outer post	
12	4	600192-99-0	Bolt MC6S 6x12 fzb	
13	4	612675-99-0	Washer, ISO 7093-1 6,4x18x1,6 fzb	
14	1	334451	Table bracket with cable clip	
15	1	1835043	Control panel, nVR2 JSM	
16	1	615904	Lock handle (A recommendation is to order lock handle if seat is repla...	
17	1	1835052	Seat	
18	1	1834989	cushion includes fasteners	
18	1	334327	Cushion, incl knobs.	
19	1	1835056	Covers for Unitrack incl. fasteners	
20	1	1836049	Joystick knob	
21	1	1836050	Gas spring	
22	1	615897	INDEXING PLUNGER GN 822-6-B-ST	
23	1	615894	INDEXING PLUNGER GN 717-6-M10-BK-ST	
24	1	1835054	Knob and plungers	

5
1-4

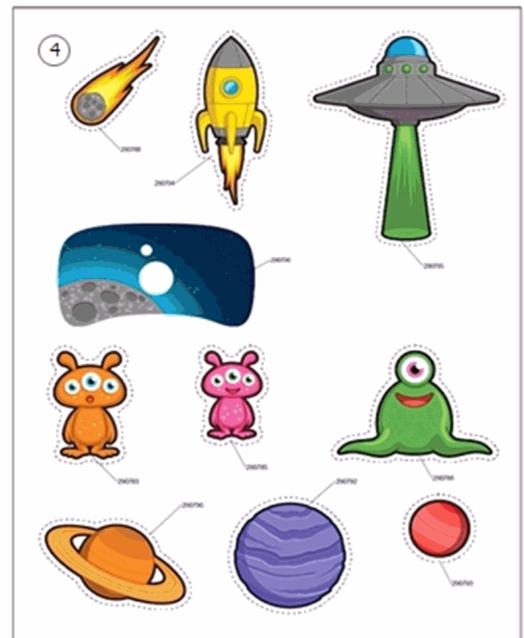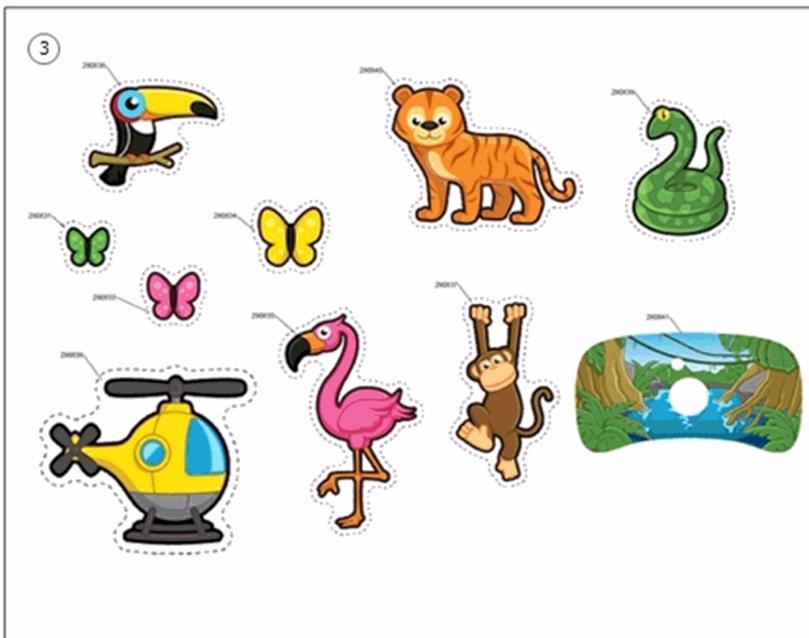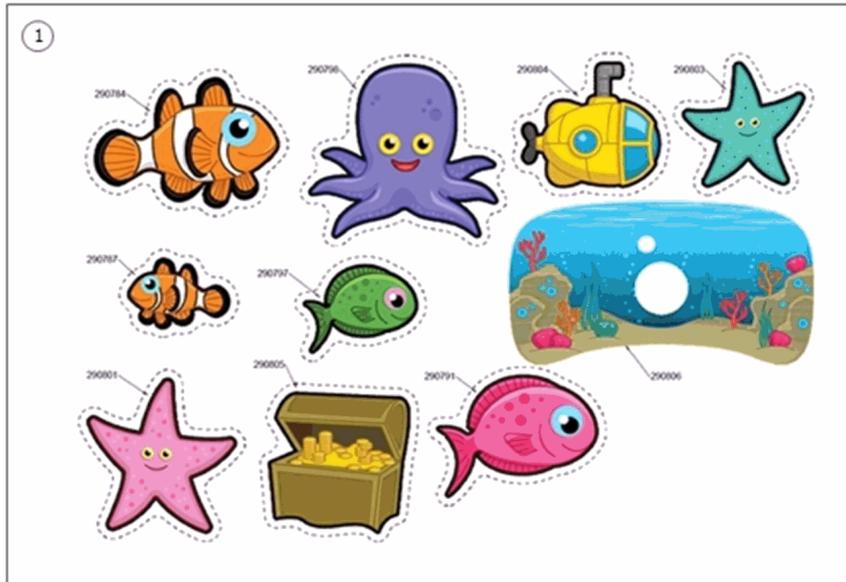

Pos	Qty	Stock Code	Description	Remark
1	1	334689	Stickers, theme ocean	
2	1	334692	Stickers, theme explorers	
3	1	334694	Stickers, theme jungle	
4	1	334693	Stickers, theme space	
5	1	1835057	Sticker kit	

Ordering information


When you order spare parts, the following must always be specified:

- The product model (for example, power wheelchair C300).
- The description of the part. Always use the description used in this catalog.
- The entire part number.
- The quantity.

NB! In some cases, parts appear in the spare part catalog without a part number. This means that the part cannot be ordered from Permobil. We have, however, chosen to describe it for your information. Standard components such as screws, nuts, washers and similar parts can be purchased in an ordinary shop using the information in this catalog. Orderable items are referred to as an 18-number.

Parts

A part consists of one or more parts. If a part consists of several parts, a bracket in connection with the part's item bubble is used to show the parts included in the part in the exploded views. See examples 1, 2 and 3.

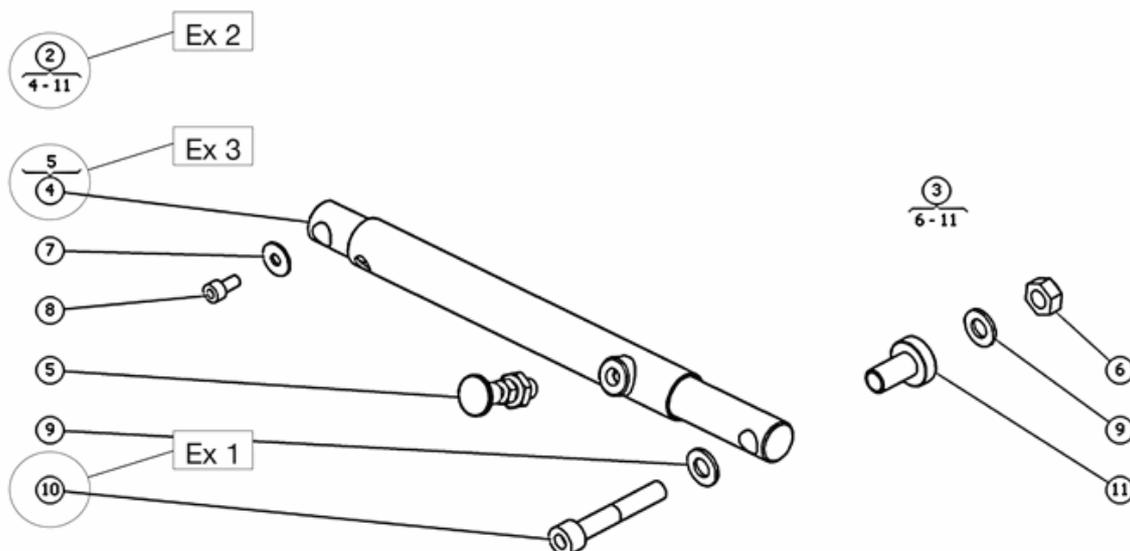

Example 1:

Position 10 refers to a single part, a screw.

Example 2:

Position 2 is a spare part consisting of the positions from 4 to 11.

Example 3:

Position 4 is an article that also includes position 5. In these cases the article or articles which referred to in the bracket, are pre-assembled on the main article (position 4).

Orderinformation


Vid beställning av reservdelar, ska följande alltid anges:

- Produkt namn/modell (till exempel, elrullstol C300).
- Benämning på delen, använd alltid benämningen som står i den här reservdelskatalogen.
- Fullständigt artikelnummer.
- Antal.

OBS! Vissa positioner i katalogen är utan artikelnummer. Det betyder att den delen inte går att beställa separat från Permobil, men vi har ändå valt att beskriva delen som information till er. Standard komponenter så som skruvar, muttrar, brickor och liknande standard komponenter kan köpas från andra leverantörer med hjälp av informationen i den här katalogen. Beställningsbara artiklar benämns med ett 18-nummer.

Reservdelar

En reservdel kan bestå av en eller flera delar. Om en reservdel består av flera delar, är positionens innehåll markerat i en klammer i anslutning till positionsbubblan. Se exemplen 1, 2 och 3 nedan.

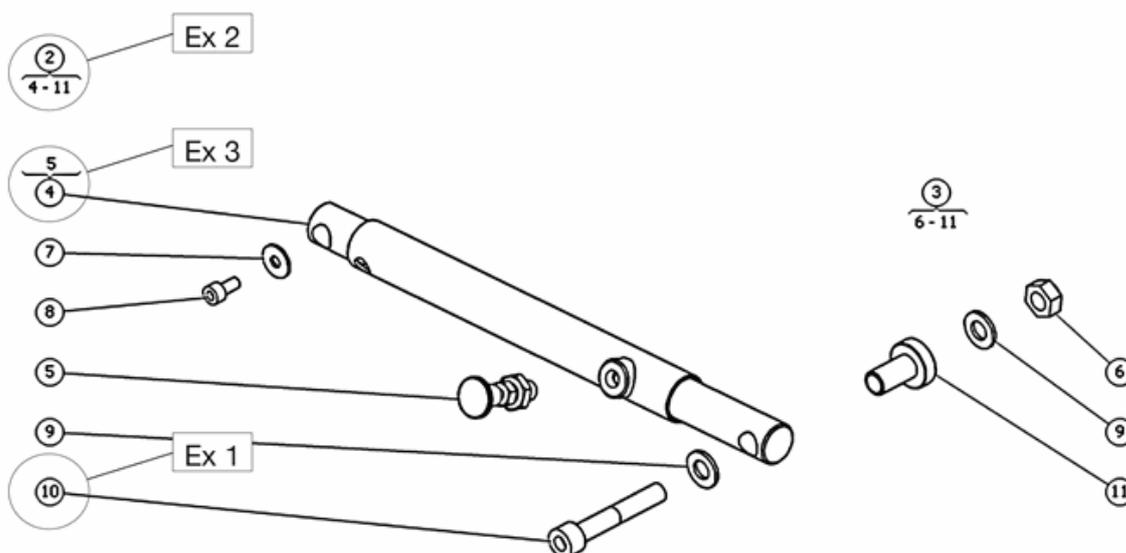

Exempel 1:

Position 10 avser en separat detalj, en skruv.

Exempel 2:

Position 2 är en reservdelssats, bestående av positionerna 4 till 11.

Exempel 3:

I position 4 ingår även position 5. I de här fallen är delen eller delarna som står i klammetet förmonterade på huvudartikeln (position 4).

CONTACT PERMOBIL

KONTAKTUPPGIFTER PERMOBIL

Permobil AB

Box 120
861 23 Timrå
Sverige
Tel: 060-59 59 00
Fax: 060-57 52 50
E-post: info@permobil.com

Permobil Inc.

300 Duke Drive
Lebanon, TN 37090
PH: 800.736.0925
FAX: 800.231.3256
info@permobil.com

Permobil AS

Industriveien 16
1481 Hagan Norge

Tlf. 815 30 077
Fax. 670 79 959
E-post: firmapost@permobil.com

Permobil Benelux BV

De Beitel 11
6466 GZ Kerkrade
Nederland
Tel: +31 (0)45 564 54 80
Fax: +31 (0)45 564 54 81
Email: info@permobil.nl



permobil