

Corpus

for C300

Seating system for Permobil power wheelchair



Corpus

Spare Part Catalog
Reservdelskatalog

CONTACT PERMOBIL:

Permobil AB

Box 120
861 23 Timrå
Sverige
Tel: 060-59 59 00
Fax: 060-57 52 50
E-post: info@permobil.se

Permobil Inc. USA

6961 Eastgate Blvd.
Lebanon, TN 37090
USA
Phone: 800-736-0925
Fax: 800-231-3256
Email: info@permobilusa.com

Permobil AS

Hvamsvingen 22
2013 Skjetten
Norge
Tlf. 48 12 90 00
Fax. 63 84 48 50
E-post: firmapost@permobil.no


Permobil Europe BV

De Doom 22
NL-6419 CX Heerlen
Netherlands
Tel: +31 (0)45 564 54 90
Fax: +31 (0) 45 564 54 91
Email: europe@permobil.nl


Produced and published by Permobil AB, Sweden
Edition no. 1, 2009-02
PAB: 1261-1

Contents	Page
Seat frame.....	6 - 7
Seat mounting kit	8 - 9
Seat extensions	10 - 11
Tilt 45°	12 - 13
Backrest	14 - 15
Backrest recline options	16 - 17
Armrest	18 - 19
Panelholders	20 - 21
Cushions	22 - 23
Legrest	24 - 25
Legrest recline options	26 - 27
R-net Electronics.....	28 - 29
Ordering Information	30 - 31


Innehåll	Sida
Sitsram	6 - 7
Sitsmonteringssats	8 - 9
Sitsförstoring/förkortning	10 - 11
Sitslutning 45°	12 - 13
Ryggstöd	14 - 15
Ryggstöds regleringar	16 - 17
Armstöd	18 - 19
Panelhållare	20 - 21
Dyner	22 - 23
Benstöd	24 - 25
Benstöds regleringar	26 - 27
R-net elektronik	28 - 29
Beställningsinformation	32 - 33

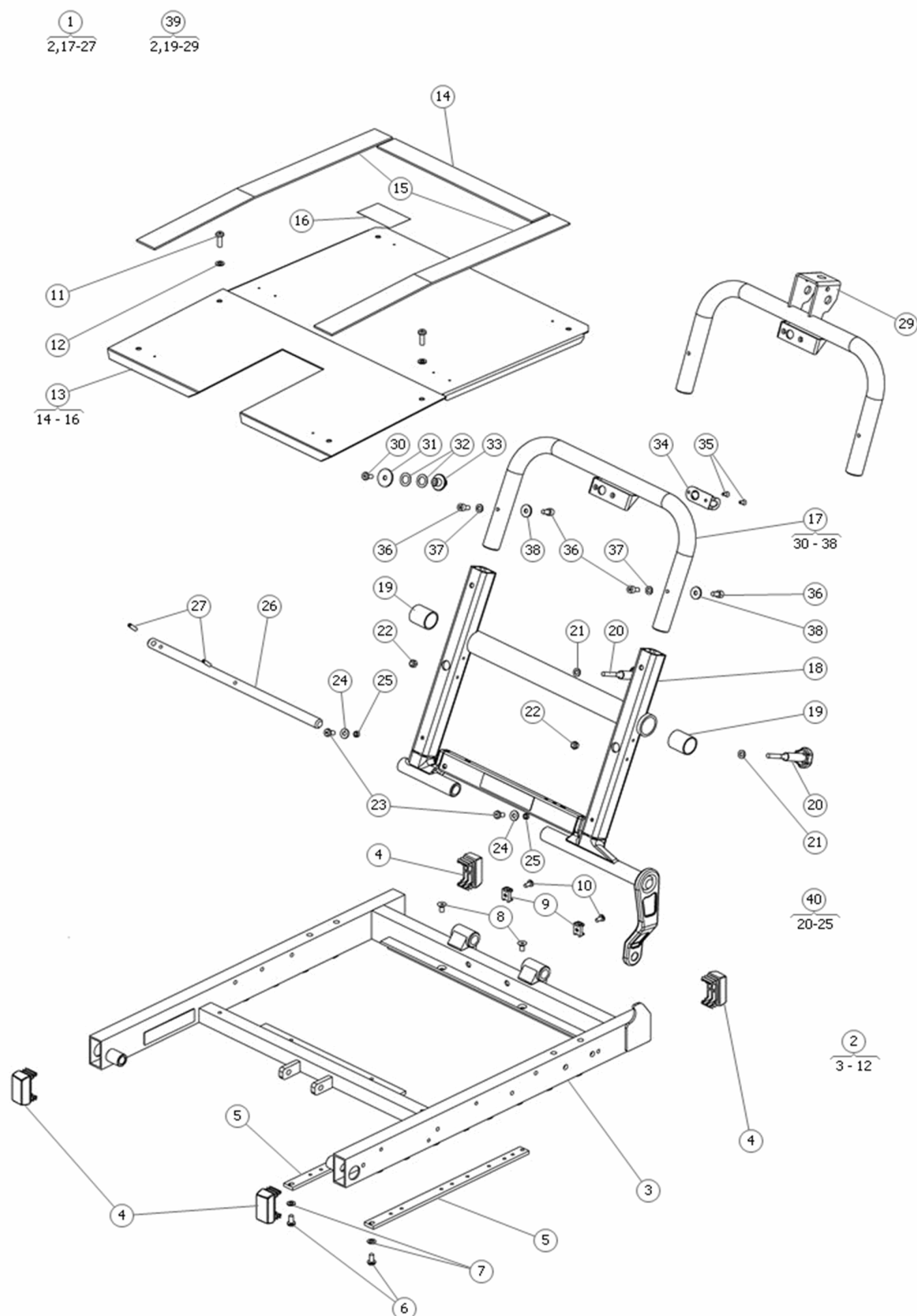

Pos	Qty	Stock Code	Description	Remark
1	1	305500-28-UD-0	SEAT CII FRAME KIT	
2	1	301573-28-0	SEAT FRAME CII	
2	1	301573-38-0	USE 301573-28-0	
3	1		FRAME	
4	4	1821420	PLASTIC PLUG SEAT FRAME	
5	2		BRACKET	
6	2	601353-99-0	BOLT K6S 5x10 fzb	
7	2	600237-99-0	WASH M5 FLAT	
8	1	601708-99-0	BOLT M5 x 10mm FHCS	
9	2	601204-99-0	CABLE TIE HOLD TM 2 S8	
10	2	601436-99-0	BOLT MRX-TT-Z 4x8 fzb	
11	2	601170-99-0	BOLT M5 x 16mm BHCS	
12	2	600237-99-0	WASH M5 FLAT	
13	1	305188-28-0	SEAT PLATE MOVEABLE	
13	1	305188-38-0	USE 305188-28-0	
14	1	600587-99-0	Velcro 30mm 1 foot length	Length 1060 mm
15	2	600587-99-0	Velcro 30mm 1 foot length	Length 1060 mm
16	1		DEKAL	
17	1	306072-28-0	BACKR REINFORCEM V.2	
18	1	305196-28-0	BACKREST HINGE CII SILVER	Use for Pilot+ & earlier chairs.
18	1	1822200	BACKREST HINGE CII R-NET	Use for R-net & VR2 chairs.
19	2	601844-99-0	SLIDE BEAR 202325 A	
20	2	306387-00-0	STAR HANDLE BACKR HINGE	
21	2	600842-99-0	WASH BRB 5.3x12x 1 fzb	
22	2	600226-99-0	LOCK NUT M5 FZB	
23	2	601353-99-0	BOLT K6S 5x10 fzb	
24	2	600237-99-0	WASH M5 FLAT	
25	2	306081-99-0	SPACER 3 o 7/5 ", EACH	
26	1	302521-99-0	SHAFT BACKR HINGE CII	
27	2	601124-99-0	EXPANDER PIN FRP 4x16	
29	1	309016-28-0	BACKR HINGE MID.MOUNT CII	For use with Co-Pilot
30	1	601353-99-0	BOLT K6S 5x10 fzb	
31	1	601935-99-0	WASH SRKB 5.5x22x1.5	
32	2		SHIM WASHER 10x16x0.5 DIN 988	
33	1	306068-99-0	QUICK LOCK PIN BACK CII	
34	1	601375-99-0	QUICK LOCK BACK HV	
35	2	601972-99-0	DUMMY RIVET TAP/K 44 BS	
36	4	600189-99-0	BOLT MC6S 5x10 fzb	
37	2	600237-99-0	WASH M5 FLAT	
38	1	612857-99-0	WASHER ISO 7093-1 5 200	
39	1	309040-28-UD-0	SEAT CII FRAME KIT	
40	1	306790-99-0	BOLT KIT BACK CII	

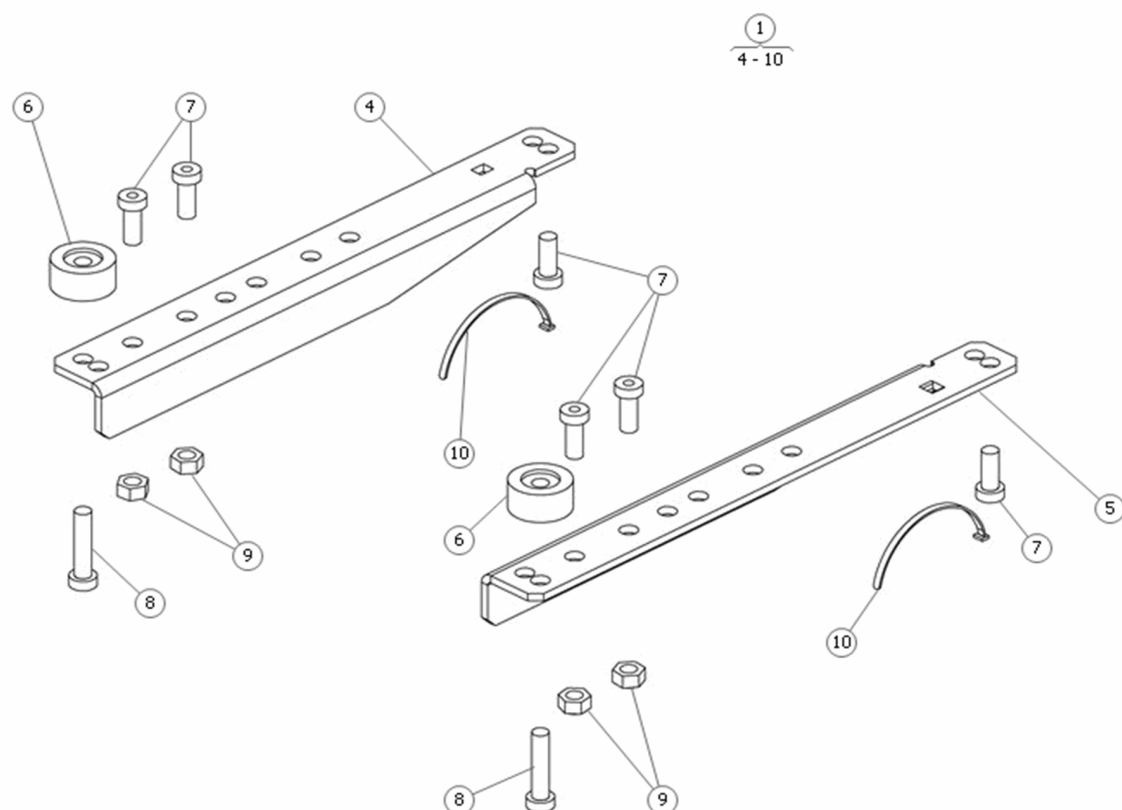


Pos	Qty	Stock Code	Description	Remark
1	1	303595-99-0	INSTALL KIT C CII MS90	
2	1	310238-99-0	INSTAL KIT LOW CH2k, C300	
3	1	310239-99-0	INSTALL KIT HIGH	
4	1		PLATE, RIGHT	
5	1		PLATE, LEFT	
6	2		SPACER Ø30/8.2/15	
6	2		SPACER	
7	6		USE 601450-99-0 BOLT MLC6S 8x20 fzb	
7	6	601450-99-0	BOLT MLC6S 8x30 fzb	
8	2	601947-99-0	BOLT MLC6S 8x35 fzb	
9	4	600228-99-0	NYLOCK NUT M8	
10	2	600136-00-0	CABLE TIE SST1M 2.4x102	
11	4	601926-99-0	BOLT MF6S 8x50 fzb	
12	4		SPACER, 8,2x25x25	
13	4	600239-99-0	WASH BRB 8.4x16x1.6 fzb	
14	4	600228-99-0	NYLOCK NUT M8	
15	4	600744-99-0	BOLT MF6S 8x25 fzb	

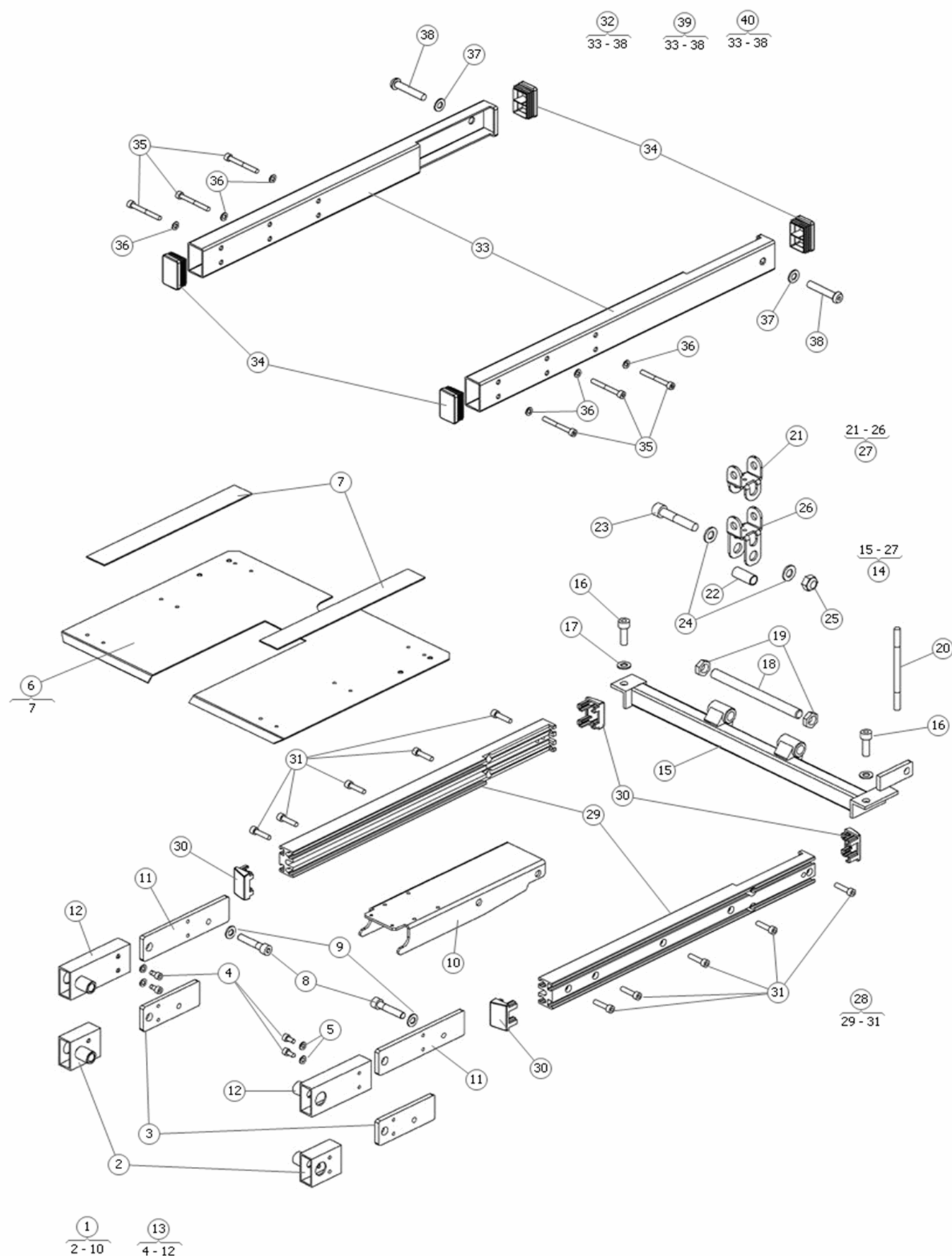


Spare Part CII for C300

Seat extensions

Pos	Qty	Stock Code	Description	Remark
1	1	305255-28-0	SEAT EXTENSION +50	
1	1	1827478	SEAT EXT+50, C300TO	THIS PART NUMBER IS ONLY USED ON THE C300..
2	2	303169-28-0	SEAT EXT SECTION +50 CII	
3	2		PLATE	
4	4	600189-99-0	BOLT MC6S 5x10 fzb	
5	4	600237-99-0	WASH M5 FLAT	
6	1	305247-28-0	SEAT PLATE EXT +50/+100	
7	2	600587-99-0	Velcro 30mm 1 foot length	
8	2	600274-99-0	BOLT MC6S 8x 40 fzb	
9	2	600239-99-0	WASH BRB 8.4x16x1.6 fzb	
10	1	305235-99-0	SLIDE BAR LEGREST C II	+50
10	1	305236-99-0	SLIDE BAR LEGREST C II	+100
10	1	305234-99-0	SLIDE BAR LEGREST CII,	USED ONLY WITH STANDARD SEAT DEPTH OF 1..
11	2		PLATE	
12	2	303172-28-0	OBSOLETE	+100
12	2	309254-28-0	SEAT EXT KIT +100 HD	+100
13	1	305260-28-UD-0	SEAT EXT KIT +100 CII	COMPLETE KIT 22"D
13	1	309254-28-0	SEAT EXT KIT +100 HD	
14	1	305265-28-0	SEAT REDUCTION -50, CII	
15	1		SEAT REDUC	
16	2	600499-99-0	BOLT MC6S 8x 25 fzb	
17	2	600239-99-0	WASH BRB 8.4x16x1.6 fzb	
18	1		BAR, M12	
19	2	600759-99-0	NUT ISO 4035 M12 fbz	
20	1	303716-99-0	BOLT PS 8x115 fzb	
21	1		BRACKET	
22	1	302330-99-0	ACTU SLEEVE 30	
23	1	601231-99-0	BOLT MC6S 10x 55 fzb	
24	2	600241-99-0	WASH BRB 10.5x20x2 fzb	
25	1	600229-99-0	LOCK NUT M10 FZB	
26	1		BRACKET	
27	1	308594-99-0	ACTU BRACK LOWER, HD3	
28	1	1822380	UT WIDTH BARS THIN PAIR	PAIR
28	1	1822381	UT WIDTH BARS -THIN PAIR	PAIR
28	1	1822382	UT WIDTH BARS- THIN PAIR	PAIR
28	1	1822383	UT WIDTH BARS -THICK PAIR	PAIR
28	1	1822384	UT WIDTH BARS- THICK PAIR	PAIR
28	1	1822385	UT WIDTH BARS -THICK PAIR	PAIR
29	2		WIDE TRACK (+50)	
30	1	1824302	END CAP FOR UT THICK RAIL	
31	10	600195-99-0	BOLT MC6S 6x25 fzb	
32	1		SEAT WIDE +50	
33	2		TUBE	
33	2		TUBE	
33	2		TUBE	
34	4	601896-00-0	PLAST PLUG VL 40x25	
35	6	602362-99-0	USE 601193-99-0	
35	6	601193-99-0	BOLT MLC6S 5x45	
36	6	600237-99-0	WASH M5 FLAT	
37	2	601884-99-0	BOLT K6S 8x45 fzb	
38	2	600239-99-0	WASH BRB 8.4x16x1.6 fzb	
39	1		SEAT WIDE +50/L=+50	
40	1		SEAT WIDE +50/L=+100	

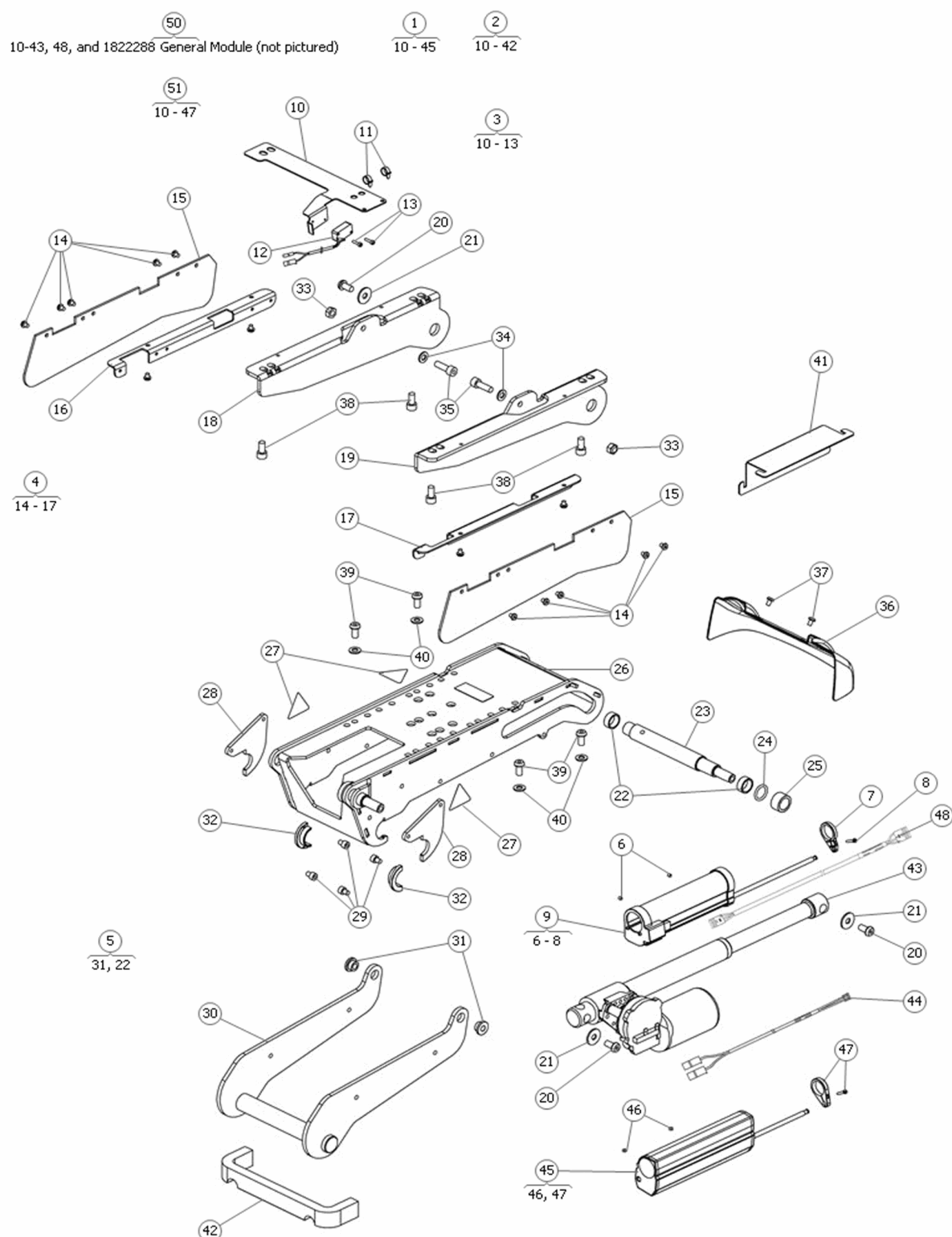


Pos	Qty	Stock Code	Description	Remark
1	1		SEAT TILT, P+	
2	1	1824003	SEAT TILT C300/C350 W/O	
3	1	314351-99-0	CABLE SET, END OF TRAVEL,	
4	1	1820432	SAFETY DEVICE TILT C300	
5	1	1825440	BEARING TILT C300, C350	
6	1	600212-99-0	SET SCREW SK6SS 4x4	
7	1	1824951	CLAMP SWITCH SOFTPOT C350	
8	1		TAPPING SCREW, 3x16 Fe/Zn	
9	1	1826368	POSITION SENSOR 150mm 30d	ICS
10	1		BRACKET	
11	2		CABLE TIE	
12	1	1824177	CABLE LOWER END STOP TILT	
13	1	601460-99-0	BOLT MC6S 3x12 fzb	
14	14	612781-99-0	BOLT K6S 5x 6 fzb	
15	2		SQUEEZE GUARD	
16	1		SQUEEZE GUARD BRACKET RIGHT	
17	1		SQUEEZE GUARD BRACKET LEFT	
18	1		SEATBRACKET RIGHT	
19	1		SEATBRACKET LEFT	
20	1	601404-99-0	BOLT K6S 8x16 fzb	
21	1	612393-99-0	Washer ISO 7093-1 Fe/Zn 8	
21	1		WASHER, 8,4x24x2 Fe/Zn ISO 7093-1	
22	2		BEARING	
22	1	1820434	REPLACED BY 1825440	OBSOLETE WHEN ZERO INVENTORY
23	1		SHAFT	
24	1		RUBBER RING	
25	1		SPACER	
26	1		BOX	
27	3	313891-99-0	DECAL TILT WARNING, "C"	
28	2		SHACKLE	
29	4	600888-99-0	BOLT MC6S 6x10 fzb	
30	1		SLEWING BRACKET	
31	1	1823932	BEARING, TILT, C300 C350	
32	2		BEARING	
33	2	600228-99-0	NYLOCK NUT M8	
34	2	600239-99-0	WASH BRB 8.4x16x1.6 fzb	
35	2	600499-99-0	BOLT MC6S 8x 25 fzb	
36	1	1820433	TILT COVER C300 (BLACK)	
37	2	600740-99-0	BOLT K6S 5x 8 fzb	
38	4	600590-99-0	BOLT MC6S 8x 16 fzb	
39	1	601404-99-0	BOLT K6S 8x16 fzb	
40	4	600239-99-0	WASH BRB 8.4x16x1.6 fzb	
41	1		PLATE	
42	1	1824097	SAFETY DEVICE FRONT COVER	
43	1	612871-99-0	ACTUATOR SEAT TILT 45°	For use with Pilot+, R-net, VR2 and VSI
44	1	305677-00-0	CABLE ACTU CONTROL	
45	1	1820435	LIMITSW KIT SEAT TILT	
46	2		SETSCREW, M4x4 DIN 916	NOT AVAILABLE
47	1	601889-99-0	CARRIER LIMIT SWITCH KIT	
48	1	1822290	CABLE SOFT POT ICS 1000mm	US Added
50	1	1824708	SEATTILT 45° SOFTPOT ICS	
51	1	1820429	SEAT TILT 45° C300, C350	

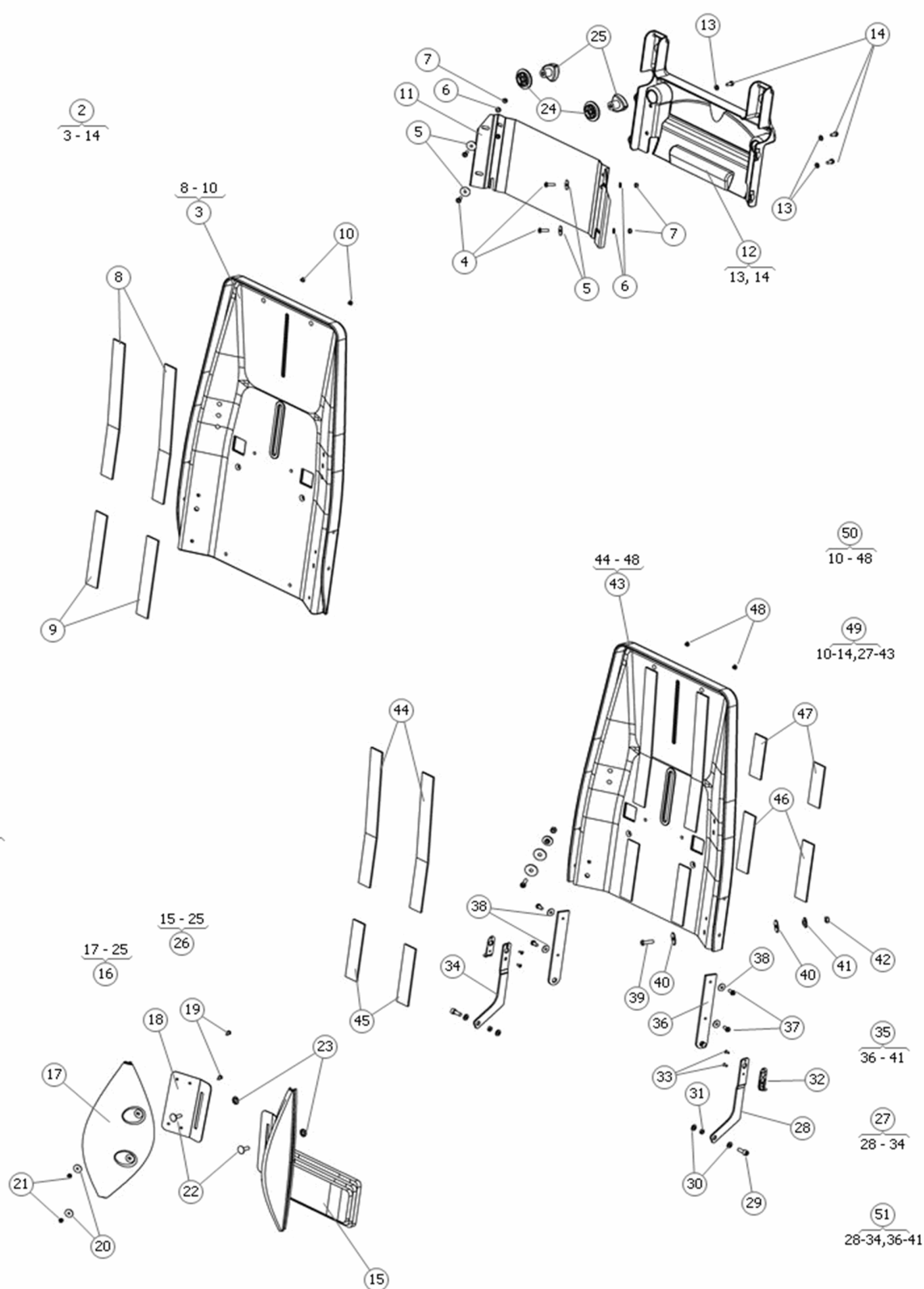


Pos	Qty	Stock Code	Description	Remark
1	1		BACKREST KIT 350x615	
1	1		BACKREST KIT 450x615	
1	1	303892-00-0	BACK SHELL, TALL BACK, +4"H	
1	1	310027-00-0	NOT USED IN USA	
1	1	310030-00-0	NOT USED IN USA	
1	1	310033-00-0	OBSOLETE-NOT USED IN USA	
1	1	305480-00-90-0	BACKR KIT FIXED CII, -50	
1	1	305300-00-90-0	REPLACED w/305300-00-UD-0	
1	1	305490-00-90-0	BACKR KIT FIXED CII, +50	
1	1	305510-00-90-0	BACKR KIT, TALL BACK +100	TALL BACKR KIT: +4" HIGH
1	1	305300-00-UD-0	BACKR KIT FIXED CII, STD.	
1	1	305490-00-UD-0	BACKREST FIXED CII, +50	
1	1	305480-00-UD-0	BACKR KIT FIXED, CII, -50	
2	1		BACKREST KIT 400x615	
2	1		BACKREST KIT 450x615	
2	1	303892-00-0	BACK SHELL, TALL BACK, +4"H	
2	1		BACKREST KIT 400x545	
2	1		BACKREST KIT 450x545	
3	1	305385-00-0	BACKR NARROW CII	
3	1	303636-00-0	BACKR SHELL ONLY, FIXED	
3	1	305545-00-0	BACKR SHELL ONLY, FIXED	
3	1	305545-15-0	OBSOLETE!	
3	1	303892-00-0	BACK SHELL, TALL BACK, +4"H	THIS IS FOR THE +4" HIGHER BACKREST KIT
3	1	310029-00-0	USED IN HOLLAND- NOT USED	
3	1	310032-00-0	BACKR SHORT CII HOLLAND	
3	1	310035-00-0	USED IN NL/HOLLAND	
3	1	305384-00-0	BACKREST SHELL ONLY, -50	
3	1	311105-00-0	BACKREST SHELL ONLY, -50	FOR SHORT BACK- 14x21
4	4	601170-99-0	BOLT M5 x 16mm BHCS	
5	4	600867-99-0	WASH NB 5x19x1 fzb	
6	4	600237-99-0	WASH M5 FLAT	
7	4	600226-99-0	LOCK NUT M5 FZB	
8	2		VELCRO TAPE L=300	
9	2		VELCRO TAPE L=170	
10	2		PLASTIC PLUG Ø9x5,2 (Ø4,4)	
11	1	305256-40-0	BACKR PLATE NARROW CII	
11	1	305251-40-0	BACKR PLATE CII, STANDARD	
11	1	305360-40-0	BACKR PLATE CII, +50 WIDE	
11	1	305199-40-0	BACKR PLATE CII, STANDARD	
11	1		BACKR PLATE SLIDE +50	
11	1	305251-40-0	BACKR PLATE CII, STANDARD	
11	1	305361-40-0	BACKR PLATE CII, +50 WIDE	
11	1	305360-40-0	BACKR PLATE CII, +50 WIDE	
12	1	306363-00-0	BACKR COV CII	
13	4	600237-99-0	WASH M5 FLAT	
14	4	601159-99-0	BOLT K6S 5x12 fzb	
15	1	306396-99-0	LUMBAR SUPP STD&+50, FOR	FOR 16" AND 18" BACKRESTS
15	1	306425-99-0	LUMBAR SUPP NARROW, -50	FOR 14" BACKREST
16	1	306400-00-0	LAT SUPP V.2, STD. 16"	
16	1	306424-00-0	LAT SUPP V.2, WIDE, 18"	
17	1		LAT SUPP PLASTIC HVII VII VIII CII	
17	2		LAT SUPP PLASTIC HVII VII VIII CII'	
18	1		BRACKET LAT.SUPP V.2 CII HVII VII VIII	
18	2	306405-99-0	USE 306424-00-0	
19	4	601187-99-0	BOLT M4 x 10mm BHCS	
20	4	601974-99-0	WASH BRB 4.3x16x 1 fzb	
21	4	600225-99-0	LOCK NUT M4 FZB NYLOCK	
22	2	601882-99-0	BOLT MVBf 6x20 H fzb	
23	1	601386-99-0	LOCK WASH S-LOCK 6x15.3	
24	2		WASH LAT SUPP V.2 CII HV V VIII	
25	2	601943-99-0	STAR HANDLE VTR.40 B-M6	
25	2	306387-00-0	STAR HANDLE BACKR HINGE	for the new cover (electronics inside)
26	1		ERGO KIT 400	
26	1		ERGO KIT 480	
27	1		INSTALL ASSEMBLY	For right and left side
28	1		PLATE LEFT	
29	2	600743-99-0	BOLT MC6S 6x16 fzb	
30	4	600238-99-0	WASH BRB 6.4x12x1.5 fzb	
31	2	304815-99-0	BUSHING Y 9	
32	2	601375-99-0	QUICK LOCK BACK HV	
33	4	601972-99-0	DUMMY RIVET TAP/K 44 BS	
34	1		PLATE RIGHT	
35	1		INSTALL ASSEMBLY	For right and left side
36	2		LOCK	
37	4	601159-99-0	BOLT K6S 5x12 fzb	
38	1	612857-99-0	WASHER ISO 7093-1 5 200	
39	2	601370-99-0	BOLT K6S 6x20 fzb	
40	4	601939-99-0	WASHER 7 Fe/Zn 5 C1	
41	2	305268-99-0	SPACER LOWER BACKR SLIDE	
42	2	600227-99-0	LOCK NUT M6 LOW fzb	
43	1	305191-00-0	BACKR SHELL ONLY, SLIDING	SHELL ONLY 16" SLIDING BACK
43	1	305389-00-0	BACKR SHELL ONLY, SLIDING	SHELL ONLY 18" SLIDING BACK
43	1	311107-00-0	BACKREST SHELL ONLY, STD.	Height = 410 mm, standard, sliding back.


Pos	Qty	Stock Code	Description	Remark
43	1	311109-00-0	BACKREST SHELL ONLY, +50	Height = 410 mm, wide, sliding back.
44	2		VELCRO TAPE L=300	
45	2		VELCRO TAPE L=125	
46	2		VELCRO TAPE L=125	
47	2		VELCRO TAPE L=90	
48	2		PLASTIC PLUG Ø9x5,2 (Ø4,4)	
49	1		BACKREST KIT 400x545	
49	1		BACKREST KIT 450x545	
50	1	305200-00-UD-0	BACKR KIT SLIDE CII, STD.	
50	1	305200-00-80-0	BACKR KIT SLIDE CII, STD.	Includes cushion- not pictured
50	1	305200-00-90-0	BACKR KIT SLIDE CII, STD.	Includes cushion- not pictured
50	1	305252-00-UD-0	BACKR KIT SLIDE CII, +50	
50	1	305252-00-80-0	BACKR KIT SLIDE CII, +50	Includes cushion- not pictured
50	1	305252-00-90-0	BACKR KIT SLIDE CII, +50	Includes cushion- not pictured
51	1	1828886	INSTALL KIT BACKR SLIDE	

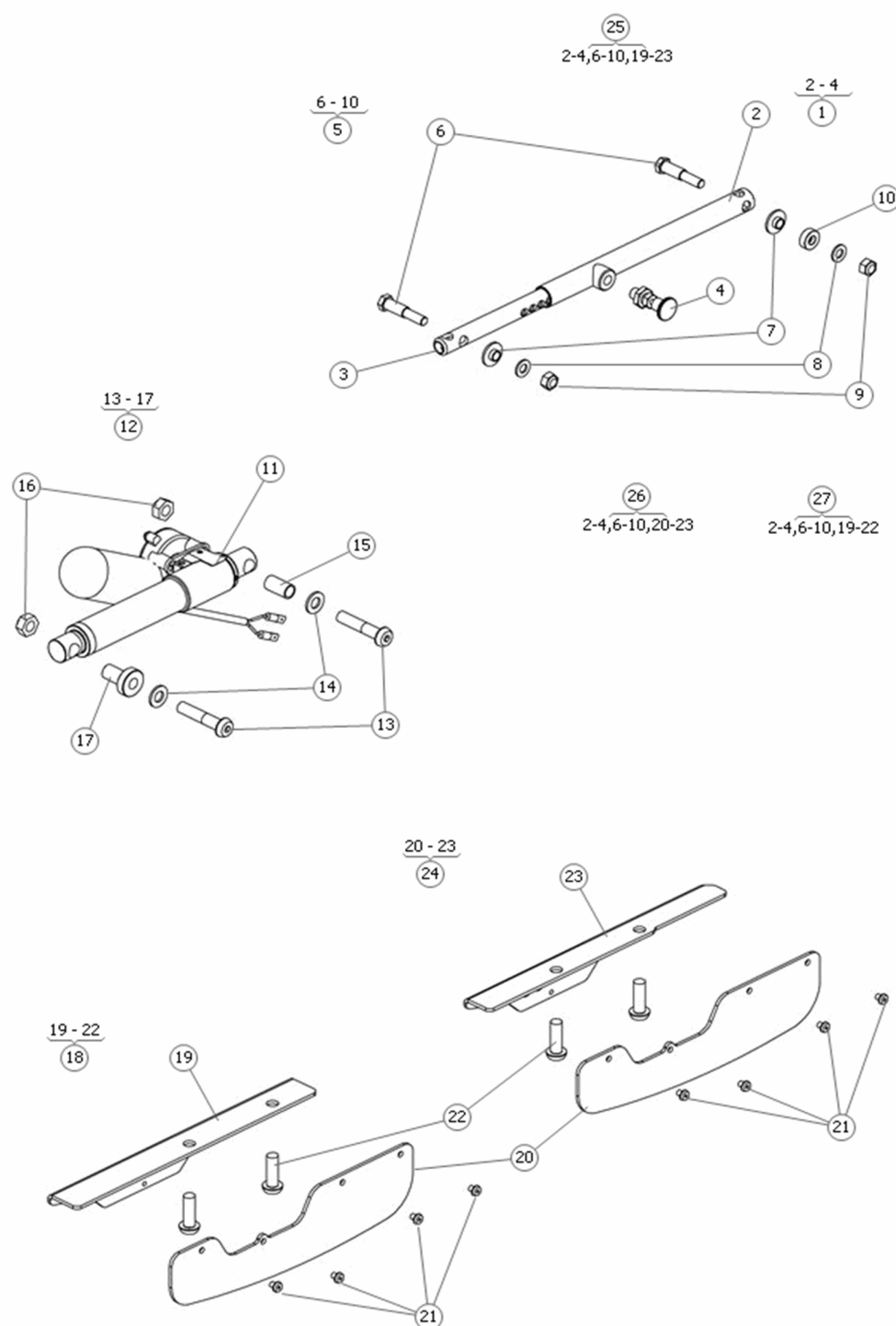


Pos	Qty	Stock Code	Description	Remark
1	1	312853-40-0	TUBE ADJUSTMENT 100MM	
2	1		OUTER TUBE	
3	1		INNER TUBE	
4	1	602328-00-0	NON-INDEXING PLUNGER	
5	1	1825154	INSTALL ASSY MAN CTRL	
6	2		LINK BOLT	
7	2		BUSHING	
8	2	600239-99-0	WASH BRB 8.4x16x1.6 fzb	
9	2	600228-99-0	NYLOCK NUT M8	
10	1		SPACER	
11	1	612805-99-0	ACTU RE25.4.100.M1.C31	
11	1	1823438	POWER CONTROL BACK- RNET	
12	1	1823437	BOLT KIT CTRL PWR BACK	
13	2		SCREW ISO 7380 M10x50 Fe/Zn	
14	2	600241-99-0	WASH BRB 10.5x20x2 fzb	
15	1	1825219	SPACER ACTU L=24 O13x1.25	
16	2	600229-99-0	LOCK NUT M10 FZB	
17	1	1827729	BUSHING, ACTUATOR	
17	1		BUSHING	
18	1	1822386	BRACKET RECLINE, RNET CII	
19	1		BRACKET	
20	1		SQUEEZE GUARD	
21	4	612781-99-0	BOLT K6S 5x 6 fzb	
22	1		BOLT K6S 10x30 fzb	
23	1		BRACKET	
24	1	1822386	BRACKET RECLINE, RNET CII	
25	1	1824609	CTRL MAN BACK INCL EL-KIT	Incl. electronics for recline control
25	1	1820538	CTRL MAN BACK CII C300	As described on this catalog page
26	1	1820538	CTRL MAN BACK CII C300	
27	1	1824609	CTRL MAN BACK INCL EL-KIT	In addition to the items shown on this page, this recl...

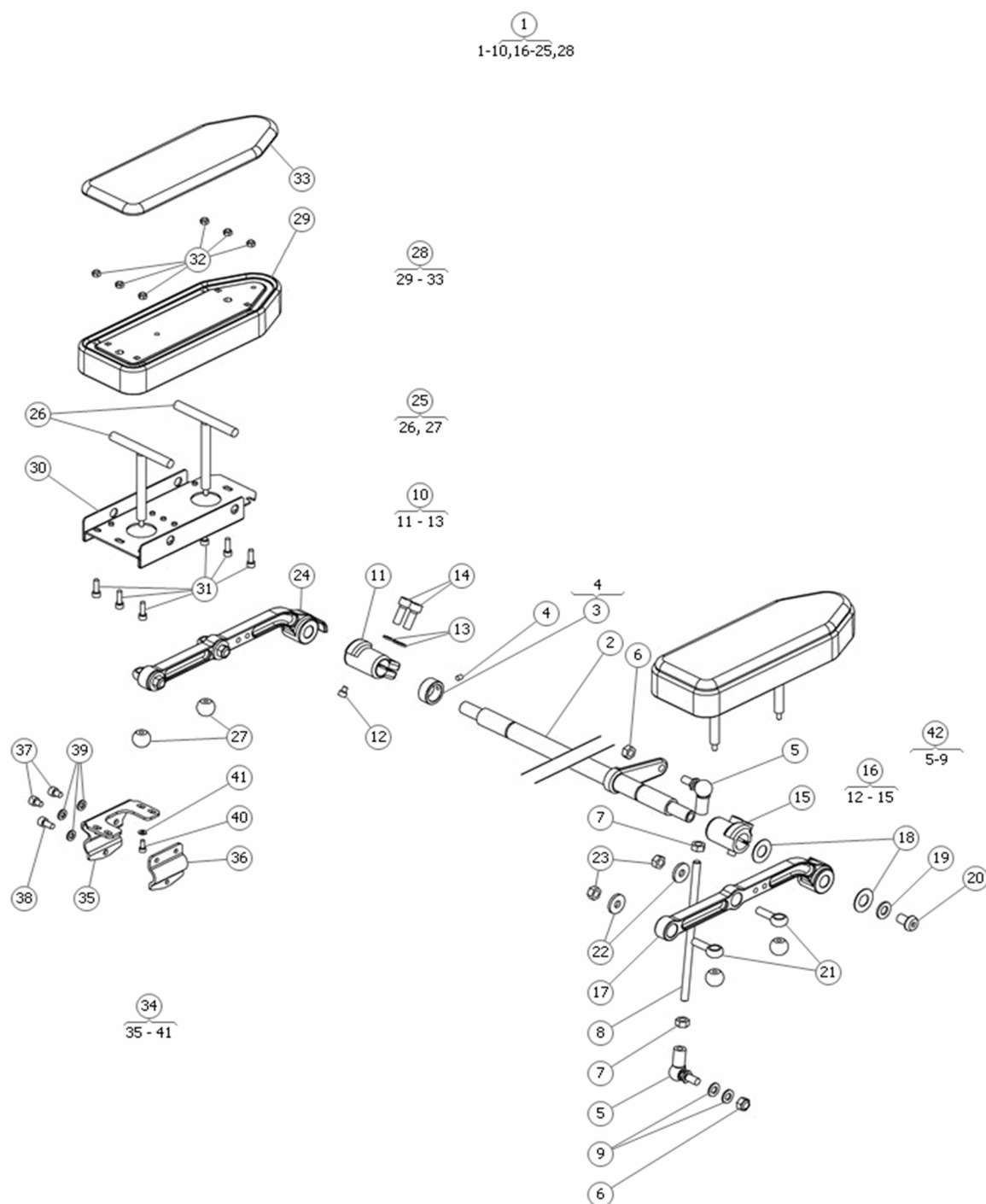


Pos	Qty	Stock Code	Description	Remark
1	1	305618-28-0	ARMREST KIT PLG +50 CII	
1	1	305267-28-0	ARMREST KIT PLG CII	
1	1	305267-38-0	USE 305267-28-0	
1	1	305267-28-0	ARMREST KIT PLG CII	
2	1	304968-99-0	RAIL ARM BRACK PLG 18"	
2	1	302722-99-0	RAIL ARM BRACK PLG 16"	
2	1	1826388	RAIL ARMREST PLG 20"	
3	1	301658-99-0	STOP COLLAR ARM C CII HV	
4	1	600546-99-0	SET SCREW M5x6 FLAT BOTTM	
5	2	601415-99-0	ANGLE LINK AS M 8 fzb	
6	2	600228-99-0	NYLOCK NUT M8	
7	2	600233-99-0	NUT M6M 8 BHB fzb	
8	1	305458-99-0	BOLT PS 8x175 fzb	
9	2	600239-99-0	WASH BRB 8.4x16x1.6 fzb	
10	1	1822340	CLAMP RING ARM RIGHT CII	
11	1		CLAMP RIGHT	
12	2	601126-99-0	BOLT MC6S 5x 6 fzb	
13	4	601427-99-0	WASH BRB 8.4x15x1.6 fzb	
14	4	601127-99-0	BOLT MC6S 8x 22 fzb	
15	1		CLAMP LEFT	
16	1	1822339	CLAMP RING ARM LEFT CII	
17	1	1820250	ARM BRACK PLG V.2	
18	4	600700-99-0	CUP SPRING 28x14.2x1	
19	2	600241-99-0	WASH BRB 10.5x20x2 fzb	
20	2	600701-99-0	BOLT K6S 10x16 fzb	
21	4	303003-99-0	LINK BOLT PLG ARM	
22	4	305390-99-0	WASH 8.5x23x3 fzb	
23	4	600228-99-0	NYLOCK NUT M8	
24	1	1820249	ARM BRACK PLG V.2	
25	1	306025-99-0	T-BRACK ARM HEIGHTENING	
25	1	1824007	T-BRACK SHORT/STD,ARMREST	
26	1		BRACKET LONG	
26	1	1824007	T-BRACK SHORT/STD,ARMREST	
27	4	601395-00-0	HANDLE KNOB 20 M5	
28	2	1822782	ARMR BLK FABRIC 10" (250	FABRIC
28	2	1822786	ARMR LEATHERETTE 10" (250	SKAI
28	2	1822783	ARMR BLK FABRIC 12" (300	FABRIC
28	2	1822787	ARMR LEATHERETTE 12" (300	SKAI
28	2	1822784	ARMR BLK FABRIC 14" (350	FABRIC
28	2	1822788	ARMR LEATHERETTE 14" (350	SKAI
28	2	1822785	ARMR BLK FABRIC 16" (400	FABRIC
28	2	1822789	ARMR LEATHERETTE 16" (400	SKAI
29	1		ARMREST 250	
29	1		ARMREST 300	
29	1		ARMREST 350	
29	1		ARMREST 400	
30	1	1822153	ARMR GUIDE PLATE, 10-12"	
30	1	1822068	ARMREST GUIDE PLATE 14"	
30	1	1821918	ARMREST GUIDE PLATE 16"	
31	6	601165-99-0	BOLT MC6S 5x16 fzb	
32	6	600226-99-0	LOCK NUT M5 FZB	
33	1	1825297	ARMR PAD BLK FABRIC 10"	FABRIC
33	1	1825301	ARMR PAD LEATHERETTE 10"	SKAI
33	1	1825298	ARMR PAD BLK FABRIC 12"	FABRIC
33	1	1825302	ARMR PAD LEATHERETTE 12"	SKAI
33	1	1825299	ARMR PAD BLK FABRIC 14"	FABRIC
33	1	1825303	ARMR PAD LEATHERETTE 14"	SKAI
33	1	1825300	ARMR PAD BLK FABRIC 16"	FABRIC
33	1	1825304	ARMR PAD LEATHERETTE 16"	SKAI
34	1	302719-99-0	PANEL HOLD BRACK	
35	1		ANGEL	
36	1		HOLDER	
37	2	601359-99-0	BOLT MC6S 6x 8 fzb	
38	1	600192-99-0	BOLT MC6S 6x12 fzb	
39	3	600238-99-0	WASH BRB 6.4x12x1.5 fzb	
40	1	600183-99-0	BOLT MC6S 4x10 fzb	
41	1	601296-99-0	WASH M4 FLAT	
42	1	302720-99-0	TIE BAR ARM SHAFT	

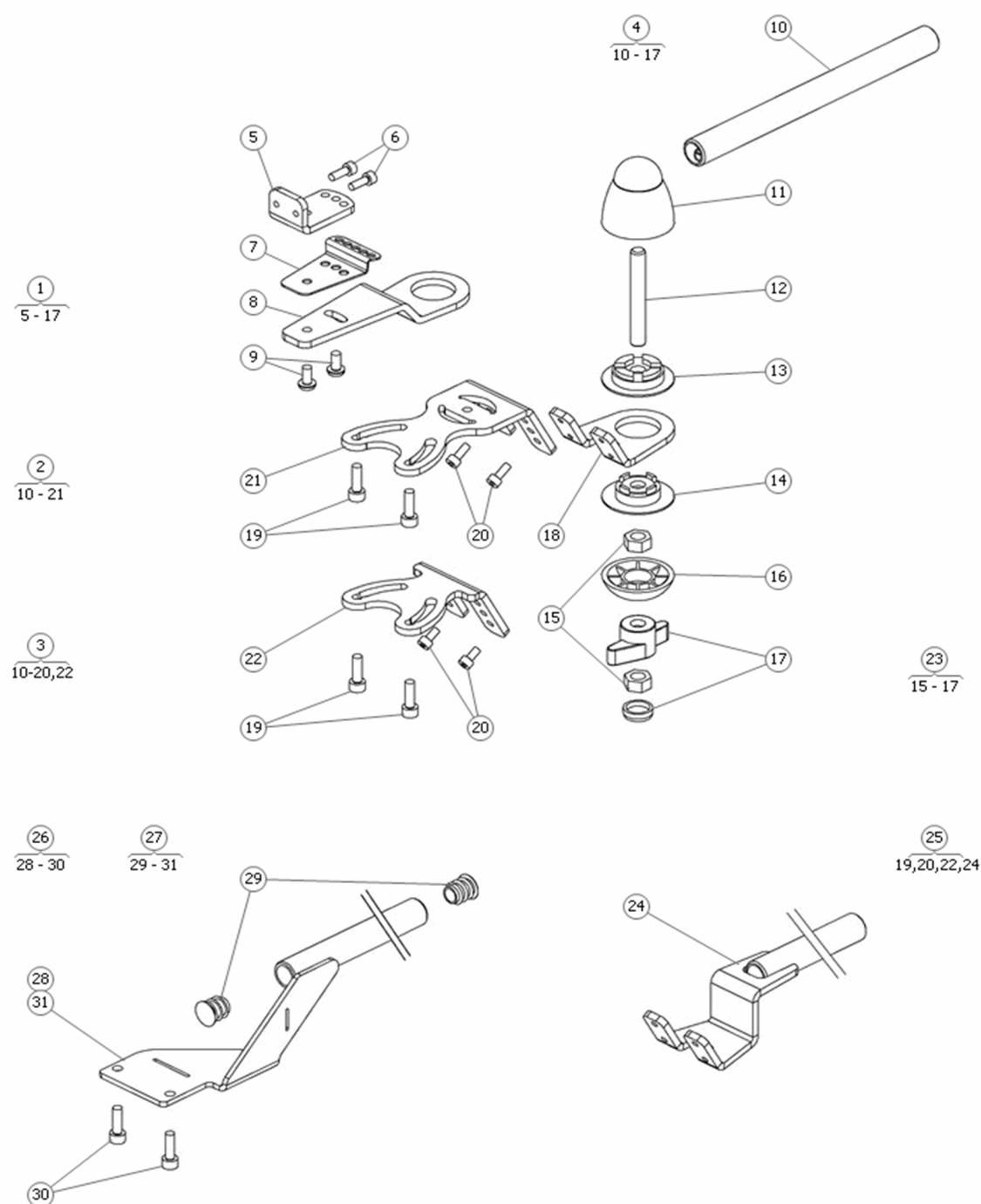


Pos	Qty	Stock Code	Description	Remark
1	1	308985-99-0	PANEL HOLD 335 HI P+	
1	1	310045-99-0	Panel Hold Extended P+	
2	1	1822455	PANEL HOLDER 335 VR2 RNET	
2	1	1822456	PANELHOLD LONG, 435, RNET	
3	1	1824242	PANEL HOLD SHORT R-NET	
4	1		HINGE L=435	
4	1		HINGE L=335	
5	1		BRACKET	
6	2	600184-99-0	BOLT MC6S 4x12 fzb	
7	1	1825189	PLATE PANELHOLDER P+, PG8	
8	1		PLATE	
9	2	601353-99-0	BOLT K6S 5x10 fzb	
10	1		TUBE L=335	
10	1		TUBE L=435	
11	1		HINGE	
12	1		SCREW, ISO 4029 M8x60 A2	
13	1		LOCK WASHER	
14	1		LOCK WASHER	
15	2	600228-99-0	NYLOCK NUT M8	
16	1		WASHER CONCAVE	
17	1		LOCK HANDLE	
18	1		PLATE	
19	2	601165-99-0	BOLT MC6S 5x16 fzb	
20	2	600183-99-0	BOLT MC6S 4x10 fzb	
21	1		PANEL HOLDER	
22	1		PANEL HOLDER V2	
23	1	1825556	ADJUSTMENT KNOB KIT	
24	1		BRACKET	
25	1	1825201	PANELHOLD, FIXED, R-NET	
26	1	1820439	PANEL HOLD VSI RIGHT	
27	1	1820440	PANEL HOLD VSI LEFT	
28	1		BRACKET, RIGHT	
29	2		PLASTIC PLUG	
30	2	601165-99-0	BOLT MC6S 5x16 fzb	
31	1		BRACKET, LEFT	

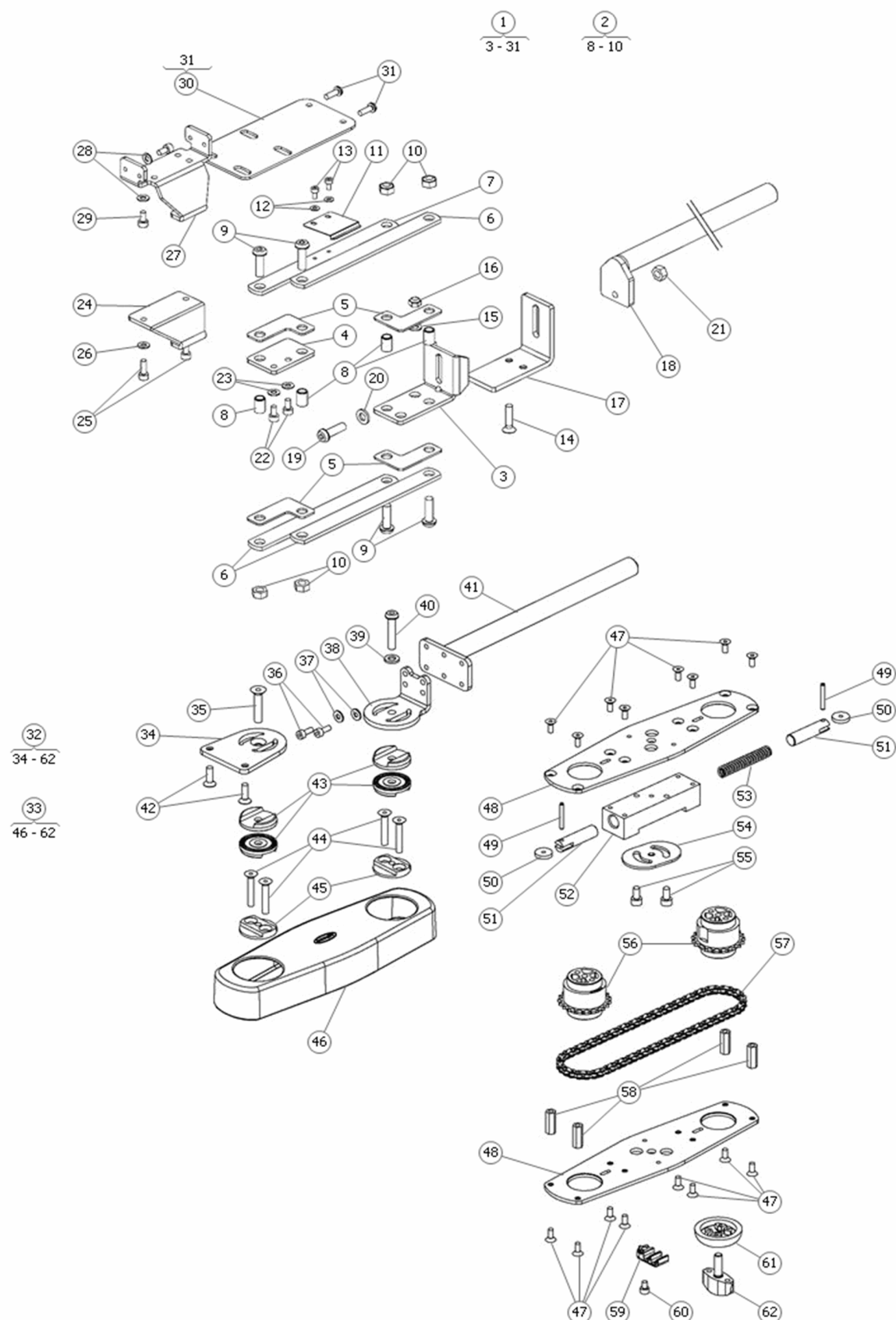


Pos	Qty	Stock Code	Description	Remark
1	1	1821782	RETRACT PANEL HOLD LEFT	Left
1	1	1826772	PANEL HOLDER PLG RIGHT	Right
2	2	1822319	REPAIR KIT, RETRACT PANEL	
3	1		PLATE	
4	1		PLATE	
5	4		PLATE	
6	3		PLATE HINGE	
7	1		PLATE HINGE	
8	2		SPACER L=12 Ø8h9/Ø6+0.2/-0	
9	2	601370-99-0	BOLT K6S 6x20 fzb	
10	2	600227-99-0	LOCK NUT M6 LOW fzb	
11	1	306814-99-0	HOLD DOWN SPRING, PANELH.	
12	2	601202-99-0	WASH BRB 3.2x 7 fzb	
13	1	601346-99-0	BOLT MC6S 3x 6 fzb	
14	1		SCREW ISO 10642 M5x22 Fe/Zn	
15	1	600237-99-0	WASH M5 FLAT	
16	1	600226-99-0	LOCK NUT M5 FZB	
17	1		PLATE REINFORCEMENT	
18	1		PANELHOLDER	
19	1	601370-99-0	BOLT K6S 6x20 fzb	
20	1	600238-99-0	WASH BRB 6.4x12x1.5 fzb	
21	1	600227-99-0	LOCK NUT M6 LOW fzb	
22	2	600182-99-0	BOLT MC6S 4x 8 fzb	
23	2	601296-99-0	WASH M4 FLAT	
24	1		PLATE	
25	2	600184-99-0	BOLT MC6S 4x12 fzb	
26	2	601296-99-0	WASH M4 FLAT	
27	1	309958-99-0	BRACK, PANEL/SW.BOX MOUNT	
28	2	601296-99-0	WASH M4 FLAT	
29	2	600182-99-0	BOLT MC6S 4x 8 fzb	
30	1		BRACKET EXTRA PANEL	
31	2	601102-99-0	BHCS ISO 7380	
32	1	1824295	L/R RETRACTABLE JOYSTICK	
33	1		HINGE	
34	1		PLATE	
35	1	600729-99-0	BOLT MF6S 6x30 fzb	
36	2	600183-99-0	BOLT MC6S 4x10 fzb	
37	1	600741-99-0	WASH BRB 4.3x10x 1 fzb	
38	1		PLATE	
39	1	600238-99-0	WASH BRB 6.4x12x1.5 fzb	
40	1	602035-99-0	USE 601372-99-0 6x45 fzb	
40	1	601372-99-0	BOLT K6S 6x45 fzb	
41	1		PANELHOLDER	
42	1	601189-99-0	BOLT MF6S 5x16 fzb	
43	4	1823502	ADJUSTMENT DISC (TEETH)	
44	4	601424-99-0	BOLT MF6S 5x30 fzb	
45	2		PLATE	
46	1		COVER	
47	1	600208-99-0	BOLT M4 x 10mm FHCS	
48	2		PLATE	
49	2		SPRING PIN, 3x24 St ISO 8750	
50	2		WHEEL	
51	2		SHAFT	
52	1		BODY	
53	1		SPRING	
54	1		PLATE	
55	2	600189-99-0	BOLT MC6S 5x10 fzb	
56	2		CHAIN SPROCK	
57	1		ROLLER CHAIN	
58	4		HEXAGON SPACER, M4 7x20 Fe/Zn	
59	1	1827135	CLAMP, RETRACTABLE JOYST	
60	1	600832-99-0	BOLT MC6S 4x 6 fzb	
61	1		WASHER	
62	1	602360-00-0	STARHANDLE CT.47630P	

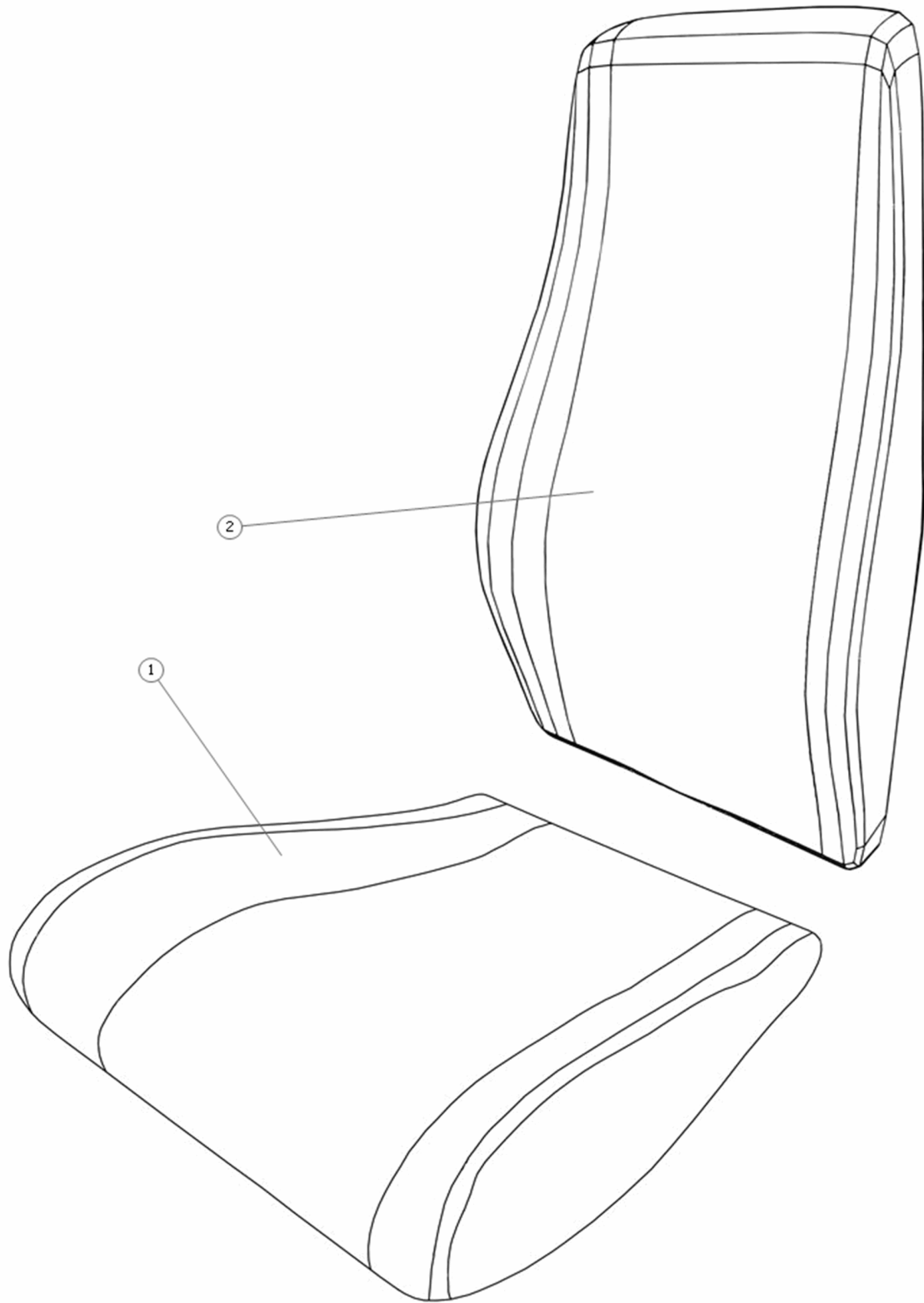


Pos	Qty	Stock Code	Description	Remark
1	1	311156-99-60-0	SEAT CUSH ERGO 420x370	
1	1	306735-99-20-0	USE 306735-99-80-0	
1	1	306735-99-60-0	SEAT CUSH ERGO 450x420	
1	1	306735-99-80-0	SEAT CUSH ERGO 450x420	
1	1	306735-99-90-0	SEAT CUSH ERGO 450x420	
1	1	306431-99-60-0	SEAT CUSH ERGO 450x500,	
1	1	306431-99-80-0	SEAT CUSH ERGO 450x500,	
1	1	306431-99-90-0	SEAT CUSH ERGO 450x500,	
1	1	306471-99-60-0	SEAT CUSH ERGO 450x560,	
1	1	306471-99-80-0	SEAT CUSH ERGO 450x560	
1	1	306471-99-90-0	SEAT CUSH ERGO 450x560	
1	1	306470-99-20-0	USE 306470-99-80-0	
1	1	306470-99-60-0	SEAT CUSH ERGO 500x500,	
1	1	306470-99-80-0	SEAT CUSH ERGO 500x500	
1	1	306470-99-90-0	SEAT CUSH ERGO 500x500	
1	1	306472-99-20-0	USE 306472-99-80-0	
1	1	306472-99-60-0	SEAT CUSH ERGO 500x560,	
1	1	306472-99-80-0	SEAT CUSH ERGO 500x560	
1	1	306472-99-90-0	SEAT CUSH ERGO 500x560	
1	1	306471-99-60-0	SEAT CUSH ERGO 450x560,	
1	1	306471-99-80-0	SEAT CUSH ERGO 450x560	
1	1	306471-99-90-0	SEAT CUSH ERGO 450x560	
1	1	310096-99-60-0	SEAT CUSH COMFORT STD	
1	1	311152-99-60-0	SEAT CUSHION COMFORT 450x	
1	1	311153-99-60-0	SEAT CUSHION COMFORT 500x	
1	1	311154-99-60-0	SEAT CUSHION COMFORT 500x	
1	1	311393-99-60-0	SEAT CUSHION COMFORT 450x	
1	1	311393-99-80-0	SEAT CUSH COMFORT 450x530	
1	1	311394-99-60-0	SEAT CUSHION COMFORT 500x	
1	1	311394-99-80-0	SEAT CUSH COMFORT 500x530	
1	1	311398-99-60-0	SEAT CUSHION 450x530 FABR	
1	1	311401-99-60-0	SEAT CUSH ERGO 500x530	
1	1	311403-99-60-0	SEAT CUSHION COMFORT 450x	
1	1	311403-99-80-0	USE 311403-99-60-0	
1	1	311494-99-60-0	SEAT CUSHION COMFORT 450X	
1	1	311494-99-80-0	USE 311494-99-60-0	
1	1	310096-99-80-0	USE 310096-99-60-0	
1	1	311398-99-90-0	SEAT CUSH ERGO 450x530	
1	1	311401-99-90-0	SEAT CUSH ERGO 500x530	
2	1		BACKR CUSH ERGO NARR CII 370x530 HOLLAND	
2	1	306475-99-60-0	Ergo Back Cush Mesh 14x26	
2	1	306475-99-80-0	BACKR CUSH ERGO 370x600	
2	1	306475-99-90-0	BACK CUSH ERGO 370x600	
2	1		BACKR CUSH ERGO STD CII VIII 420x530 HOLLAND	
2	1	306430-99-20-0	REPLACED w/306430-99-80-0	
2	1	306430-99-60-0	CUSHION BACKREST 16x25.5	
2	1	306430-99-80-0	CUSHION BACKREST 16x25.5	
2	1	306430-99-90-0	CUSHION BACKREST 16x25.5	
2	1		BACKR CUSH ERGO WIDE CII VIII 470x530 HOLLAND	
2	1	306473-99-20-0	USE 306473-99-80-0	
2	1	306473-99-60-0	Ergo Back Cush Mesh 18x26	
2	1	306473-99-80-0	BACKR CUSH ERGO 470x600	
2	1	306473-99-90-0	BACK CUSH ERGO, 18x26	
2	1	306430-99-80-0	CUSHION BACKREST 16x25.5	
2	1	1824854	BACKR CUSH ERGO, -50	
2	1	1824856	BACKR CUSH ERGO, STANDARD	
2	1	1824857	BACKR CUSH ERGO, +50 WIDE	
2	1	306473-99-80-0	BACKR CUSH ERGO 470x600	
2	1	311143-99-80-0	BACKR CUSH ERGO, -50	
2	1	311143-99-90-0	BACKR CUSH ERGO, NARROW	
2	1	311144-99-80-0	BACKR CUSH ERGO, STANDARD	
2	1	311144-99-90-0	BACKR CUSH ERGO, STANDARD	
2	1	311145-99-80-0	BACKR CUSH ERGO, +50 WIDE	
2	1	311145-99-90-0	BACR CUSH ERGO, +50 WIDE	

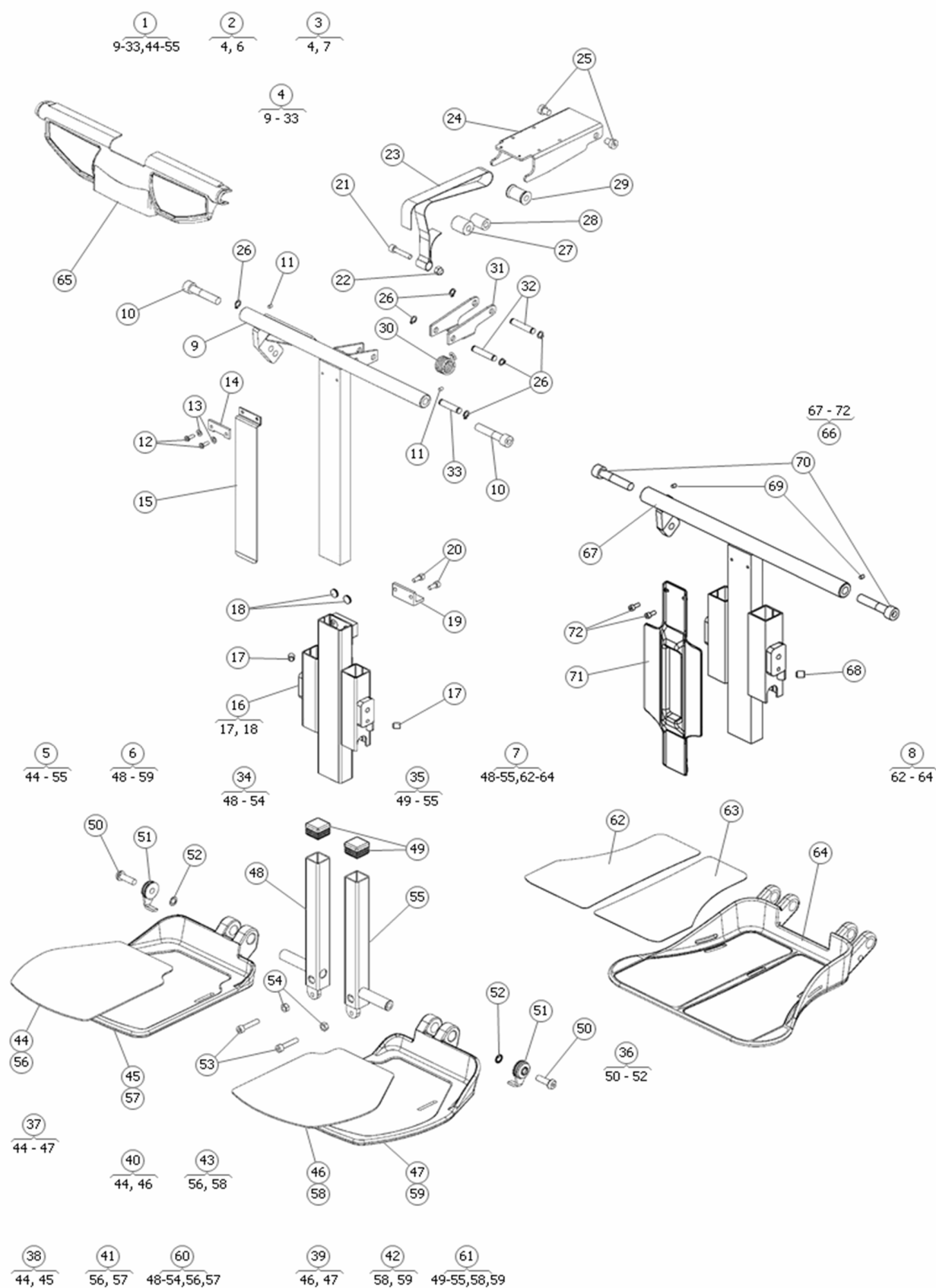

Pos	Qty	Stock Code	Description	Remark
1	1	310281-28-0	LEGR ASSEM 2PC FOOTPL,CII	
2	1		LEGREST INCL 2-PIECE FOOTPLATE NARROW	
3	1		LEGREST INCL 1-PIECE FOOTPLATE	
4	1		LEGREST	
5	1		FOOTPLATE	
6	1		FOOTPLATE	
7	1		BRACKET INCL. WHOLE FOOTPL.	
8	1	312991-28-0	FOOTPLATE ONEPIECE STREET	
9	1	310017-99-0	INNER SECT.UP HINGE LEG	PART DIFFERS FROM WHAT IS SHOWN- 310017-..
10	1	601227-99-0	BOLT MC6S 10x 50 fzb	
11	1	600556-99-0	SET SCREW M4x5 FLAT BOTTM	
12	1	601102-99-0	BHCS ISO 7380	
13	1	601296-99-0	WASH M4 FLAT	
14	1	1829669	WEB BELT HOLDER	
15	1	305183-28-0	PROTECT PLATE LEGREST CII	
16	1	305240-28-0	OUT SECT FOOTPL BRAC CII	
16	1	305240-38-0	USE 305240-28-0	
17	1	600737-99-0	SET SCREW SK6SS 8x10	
18	1	305162-99-0	SLIDE PLATE Y 12x2.50	
19	1	305237-99-0	WEB BRACK LEGREST CII GN	
20	1	600169-99-0	BOLT M5 x 12mm SHCS	
21	1	600196-99-0	BOLT MC6S 6x30 fzb	
22	1	1821261	LOCK NUT M6 HIGH fzb	
23	1	305245-00-0	WEB BELT CALF REST CII	
24	1	305234-99-0	SLIDE BAR LEGREST CII,	
25	1	601703-99-0	BOLT MLC6S 8x10 fzb	
26	1	600503-99-0	CIRCLIP SGA 8	
27	1	1821258	ROLLER O 16 mm WEB BELT	
27	1	305224-99-0	ROLL 23mm CII	
28	1	1821258	ROLLER O 16 mm WEB BELT	
29	1	1821259	ROLLER O 17 mm WEB BELT	
30	1	1821260	SPRING WEB BELT	
31	1	1821089	ROLL BRACK CII	
32	1	305221-99-0	SHAFT 43mm ROLL	
33	1	305223-99-0	SHAFT 36mm ROLL	
34	1	308343-99-1	BRACK FOOTPL LEGREST CII	
35	1	308343-99-2	BRACK FOOTPL LEGREST CII	
36	1	308371-99-0	FRICTION GUIDE FOOTPLATES	
37	1		FOOTPLATES PAIR WIDE	
38	1	1821244	FOOTPLATE, RIGHT, STANDARD	
39	1	1821243	FOOTPLATE, LEFT, STANDARD	
40	1	1820716	SLIP PROTECT SET FOOTPLAT	
41	1	1827560	FOOTPL DUAL RIGHT, GREY	Grey
41	1	1827562	USE 1827560	Black
42	1	1827561	FOOTPL DUAL, LEFT, GRAY	Grey
42	1	1827563	USE 1827561	Black
43	1	1820714	SLIP PROTECT SET FOOTPLAT	
44	1		RUBBER PROTECT	
45	1		FOOTPLATE	
46	1		RUBBER PROTECT	
47	1		FOOTPLATE	
48	1		BRACKET	
49	1	601236-00-0	PLAST PLUG 56-25-25	
50	1	602165-99-0	BOLT K6S 8x25 fzb	
51	1		FRICTIONS BRAKE	
52	1	602127-99-0	WASH SCHNORR 8.4x13x0.8	
53	1	600196-99-0	BOLT MC6S 6x30 fzb	
54	1	600232-99-0	NUT M6M 6 fzb	
55	1		BRACKET	
56	1		RUBBER PROTECT	
57	1		FOOTPLATE	
58	1		RUBBER PROTECT	
59	1		FOOTPLATE	
60	1	1824152	FOOTPL INCL. BRACKET	
61	1	1824151	FOOTPL INCL. BRACKET	
62	1		RUBBER PROTECT	
63	1		RUBBER PROTECT	
64	1		FOOTPLATE	
65	1	1821179	SQUEEZE GUARD - LEGREST	
66	1	1824899	LEGREST NO EXTENSIONS	
67	1		Legrest base TO	
68	1	600737-99-0	SET SCREW SK6SS 8x10	
69	1	600556-99-0	SET SCREW M4x5 FLAT BOTTM	
70	1	601227-99-0	BOLT MC6S 10x 50 fzb	
71	1	1825199	COVER LEGREST, C300TILT	
72	1	600183-99-0	BOLT MC6S 4x10 fzb	

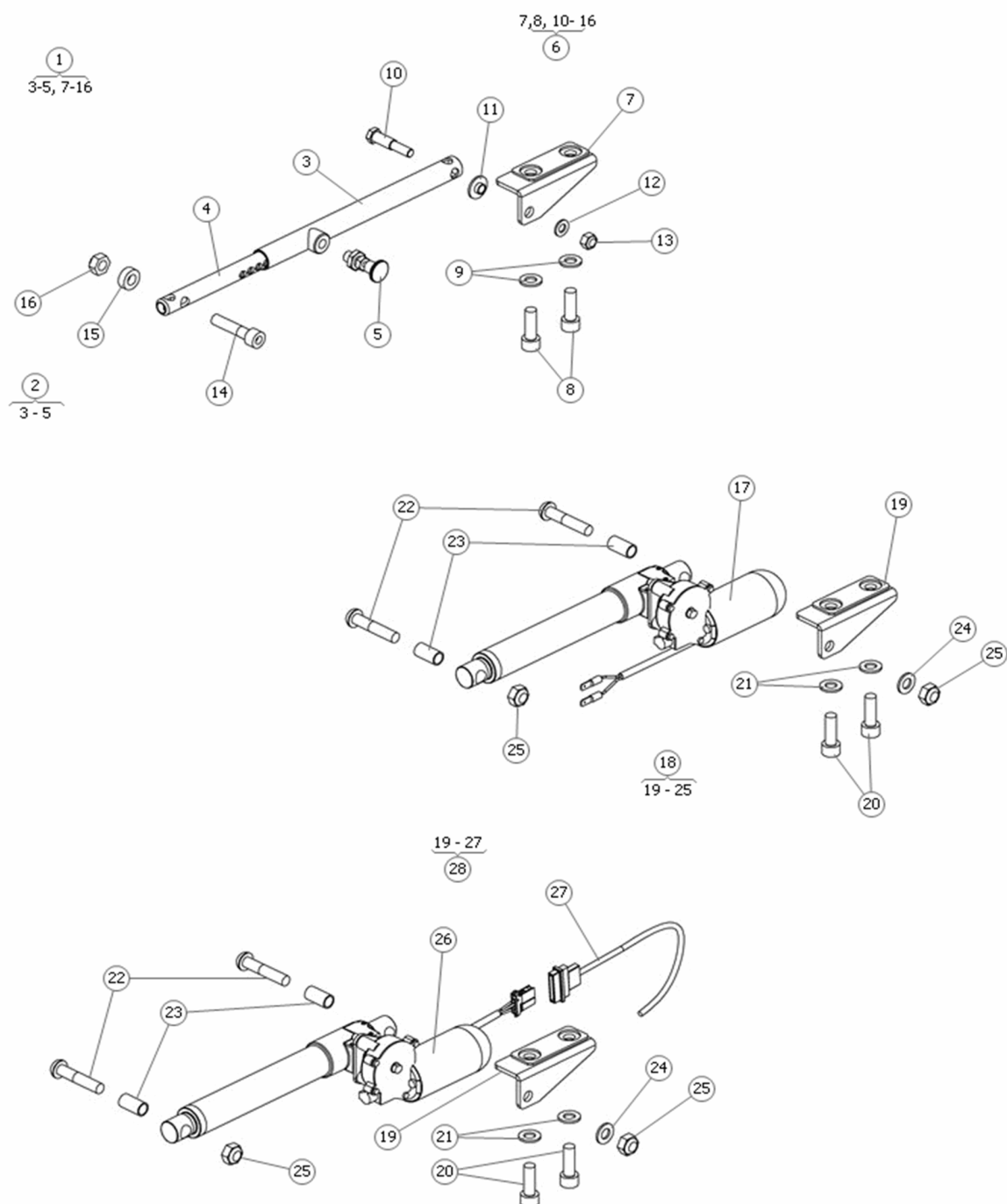

Pos	Qty	Stock Code	Description	Remark
1	1	1820537	CTRL MAN LEG CII C300/350	
2	1	312853-40-0	TUBE ADJUSTMENT 100MM	
3	1		OUTER TUBE	
4	1		INNER TUBE	
5	1	602328-00-0	NON-INDEXING PLUNGER	
6	1		INSTALL ASSEMBLY	
7	1		BRACKET	
8	2	600205-99-0	BOLT MC6S 10x 30 fzb	
9	2	600241-99-0	WASH BRB 10.5x20x2 fzb	
10	1		LINK BOLT	
11	1		BUSHING	
12	1	600239-99-0	WASH BRB 8.4x16x1.6 fzb	
13	1	600228-99-0	NYLOCK NUT M8	
14	1	600277-99-0	BOLT MC6S 10x 45 fzb	
15	1	1827732	SPACER NYLON 10.5x20x7	
16	1	600229-99-0	LOCK NUT M10 FZB	
17	1	612804-99-0	LEG ACTUATOR, C300/C350	
18	1	313824-28-0	BOLT KIT CTRL POWER LEG	
19	1		BRACKET	
20	2	600205-99-0	BOLT MC6S 10x 30 fzb	
21	2	600241-99-0	WASH BRB 10.5x20x2 fzb	
22	2		SCREW ISO 7380 M10x50 Fe/Zn	
23	2	1825219	SPACER ACTU L=24 O13x1.25	
24	1	600241-99-0	WASH BRB 10.5x20x2 fzb	
25	2	600229-99-0	LOCK NUT M10 FZB	
26	1	1823289	ACTU ICS LEGREST - C300 /	
27	1	1821533	ICS CABLE EXTENSION 500mm	
28	1	1823439	POWER CTRL LEGREST- RNET	

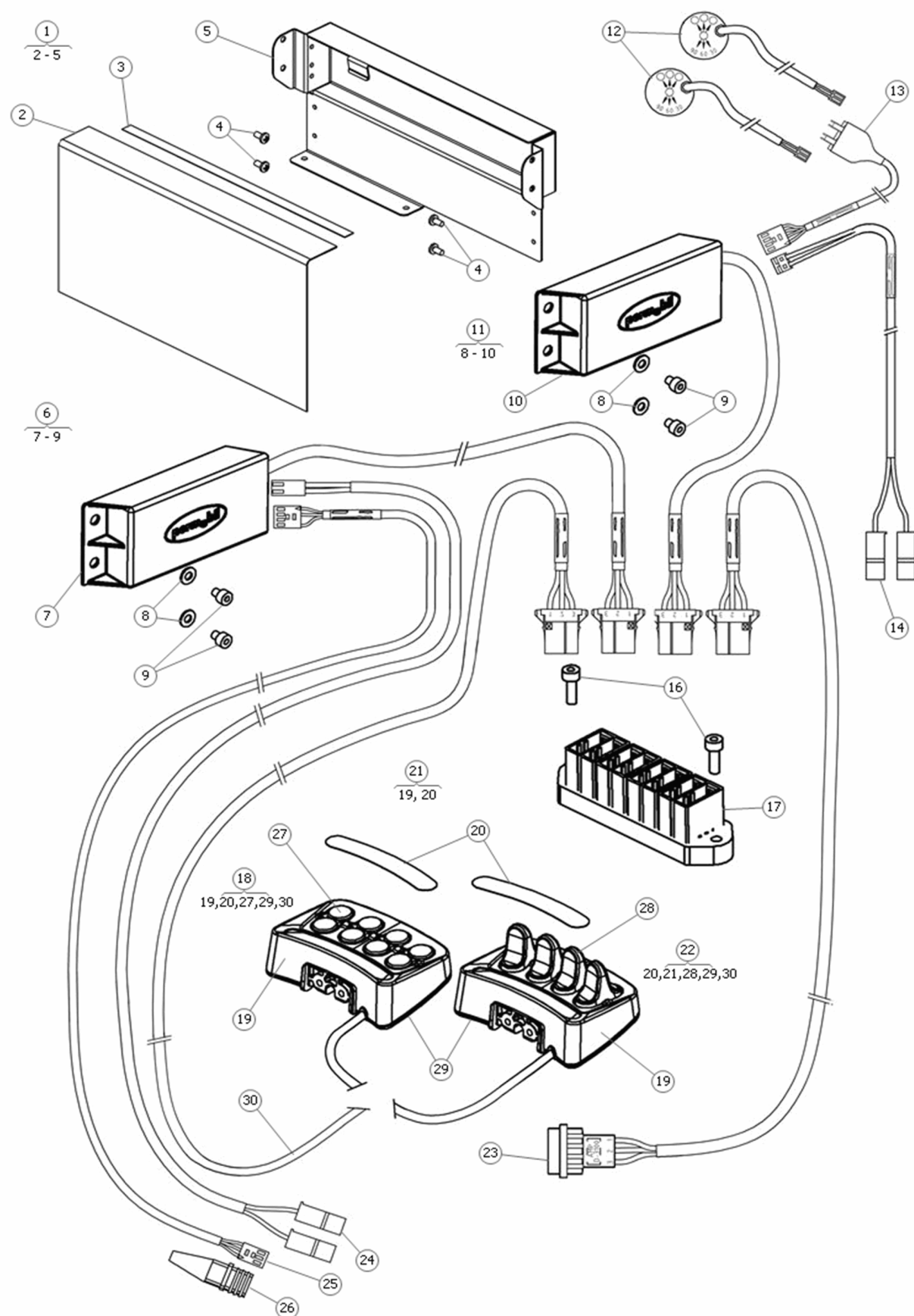

Pos	Qty	Stock Code	Description	Remark
1	1	1824805	BRACKET ELECTRONICS	
2	1		PROTECT	
3	1		ADHESIVE TRANSFER TAPE	
4	4	601436-99-0	BOLT MRX-TT-Z 4x8 fzb	
5	1		PLATE	
6	1	1822288	GENERAL MODULE ICS, TILT	
7	1		GENERAL MODULE	
8	2	601296-99-0	WASH M4 FLAT	
9	1	601282-99-0	BOLT MC6S 4x 5 fzb	
10	1		GENERAL MODULE BACKREST ICS	
11	1	1823034	GENERAL MODULE ICS,	
12	2	1822437	CABLE ANGLE STOP	
13	1	1823033	CABLE ADAPT. 2 ANGLE STOP	
14	1	306293-00-0	CABLE ACTU CONTROL L	
16	2	600184-99-0	BOLT MC6S 4x12 fzb	
17	1	1821304	8 PORT JUNCTION HUB ICS	
18	1	1823293	ICS SWITCHBOX, BUTTONS,	
19	1		SWITCH	
20	1	1822329	DECAL SET SWITCHBOX ICS	
21	1	1828839	TOP COVER ICS SWITCH BOX	Incl. sticker set
22	1	1823294	ICS SWITCHBOX, TOGGLES	
23	1	1821827	ICS CABLE EXTENSION 700mm	
24	1	305677-00-0	CABLE ACTU CONTROL	
25	1	1822290	CABLE SOFT POT ICS 1000mm	
26	1		PROTECT COVER	
27	1	1823850	PUSH BUTTON OVERLAY ICS	
28	1	1823851	TOGGLE OVERLAY ICS SWITCH	
29	1	1826515	SWITCH BOX BOTTOM ICS	
30	1	1821527	CABLE, ICS SWITCHBOX 1.4m	

When you order spare parts, the following must always be specified:

- The product model (for example, power wheelchair C300).
- The description of the part.
- always use the description used in this catalog.
- The entire part number.
- The quantity.

NB! In some cases, parts appear in the spare part catalog without a part number. This means that the part cannot be ordered from Permobil. We have, however, chosen to describe it for your information. Standard components such as screws, nuts, washers and similar parts can be purchased in an ordinary shop using the information in this catalog.

Description of Positioning list

The positioning list consist of:

A = Indicates the sparepart-group covered by the catalog.

B = Indicates part of the sparepart-group.

C = Under C there are five columns:

- 1** = Pos.no: Position number
- 2** = Pcs: Total number of given pos-no.
- 3** = Part no.: Part number
- 4** = Description: Article name
- 5** = Remark: Reserved for additional information such as reservations, restrictions etc.


Spare Part Catalog Reservdelskatalog		C300 C300		Seat Tube, Fixed Position Fast sitsrör	
1	2	3	4	5	
Pos no.	Pcs	Part no.	Description	Benämning	Remark/Anm.
1	1	1820446	Seat Tube	Sitsrör fast	
1	1	1820418	Seat Tube for tilt	Sitsrör fast för sitslutning	
2	1		Outer tube	Ytterrör	
3	1		Screw ISO 4762 M8x55 fzb	Skruv ISO 4762 M8x55 fzb	
4	4	1820421	Cap	Hätta	
5	1		Cable Tie Holder 10x15	Buntbandsfäste 10x15	
6	1		Screw ISO 7380 M4x6 fzb	Skruv ISO 7380 M4x6 fzb	
7	1		Nut DIN 985 M4 fzb	Mutter DIN 985 M4 fzb	
8	4		Screw ISO 4762 M6x10 fzb	Skruv ISO 4762 M6x10 fzb	
9	1	1820422	Inner tube	Innerrör	
9	1	1820419	Inner tube for tilt	Innerrör för tilt	
10	1		Screw ISO 4762 M6x10 fzb	Skruv ISO 4762 M6x10 fzb	
11	1		Bracket	Fjäder	
12	1		Stop	Stång	
13	1		Screw ISO 7380 M3x6 fzb	Skruv ISO 7380 M3x6 fzb	
14	1		SLS Bracket	SLS fäste	

Part

A part consists of one or more parts. If a part consists of several parts, a bracket in connection with the part's item bubble is used to show the parts included in the part in the exploded views. See example 1.

Quantity

The quantity specified in front of a part is the total quantity for the entire product.

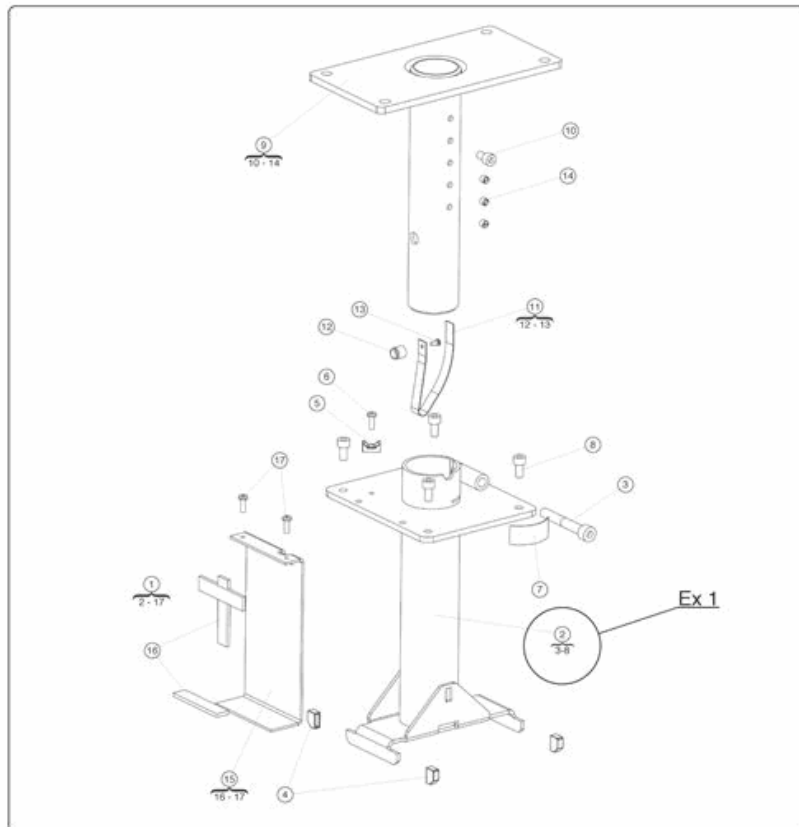**Example 1:**

The fixed seat tube, pos 2, includes pos 3 to 8, that are indicated in the bracket below (see above).

Example 2:

If a outer tube, pos 2, is ordered, the quantity is 1. In other words, there is just one outer tube on the chassis. However, if you look at the number of parts included, the quantity may vary. For example, in pos 4, the complete outer tube comprises 4 parts.

Vid beställning av reservdelar skall följande alltid anges:

- Produktmodell (ex.vis elrullstol C300).
- Artikelns benämning.
- använd samma benämning som i denna reservdelskatalog vid beställning.
- Hela artikelnumret.
- Antal.

OBS! I reservdelskatalogen förekommer i vissa fall detaljer utan angivet artikelnr. Det innebär att detaljen inte går att beställa från Permobil, men vi har ändå valt att beskriva dem för din information. Standardkomponenter så som skruv, mutter, bricka och liknande detaljer kan införskaffas i den ordinarie detaljhandeln med hjälp av informationen i denna katalog.

Beskrivning av positionslistor

Positionslistan består av:

A = Visar vilken produkt katalogen beskriver.

B = Visar vilken del av produkten som visas på aktuell sida.

C = Under C finns fem kolumner:

- 1 = Pos.no: Positionsnummer
- 2 = Pcs: Antal detaljer som ingår i aktuellt pos.nummer
- 3 = Part no.: Artikelnummer
- 4 = Benämning: Artikelns benämning (svensk och engelsk benämning)
- 5 = Anmärkning: Plats för ev. tilläggsinformation, förbehåll etc.


Spare Part Catalog Reservdelskatalog		C300 C300		Seat Tube, Fixed Position Fast sitsrör	
1	2	3	4	5	
Pos no.	Pcs	Part no.	Description	Benämning	Remark/Anm.
1	1	1820446	Seat Tube	Sitsrör fast	
1	1	1820418	Seat Tube for tilt	Sitsrör fast för sitslutning	
2	1		Outer tube	Ytterrör	
3	1		Screw ISO 4762 M8x55 fzb	Skruv ISO 4762 M8x55 fzb	
4	4	1820421	Cap	Hätta	
5	1		Cable Tie Holder 10x15	Buntbandsfäste 10x15	
6	1		Screw ISO 7380 M4x6 fzb	Skruv ISO 7380 M4x6 fzb	
7	1		Nut DIN 985 M4 fzb	Mutter DIN 985 M4 fzb	
8	4		Screw ISO 4762 M6x10 fzb	Skruv ISO 4762 M6x10 fzb	
9	1	1820422	Inner tube	Innerrör	
9	1	1820419	Inner tube for tilt	Innerrör för tilt	
10	1		Screw ISO 4762 M6x10 fzb	Skruv ISO 4762 M6x10 fzb	
11	1		Bracket	Fjäder	
12	1		Stop	Stång	
13	1		Screw ISO 7380 M3x6 fzb	Skruv ISO 7380 M3x6 fzb	
14	1		SLS Bracket	SLS fäste	

Artikel

En artikel består av en eller flera detaljer. Om en artikel består av flera detaljer är det markerat i sprängskisserna med ett klamer i anslutning till artikelns posbubbla där det står vilka detaljer som ingår i artikeln. se Ex 1.

Antal

Det angivna antalet framför en artikel är det sammanlagda antalet för hela produkten.

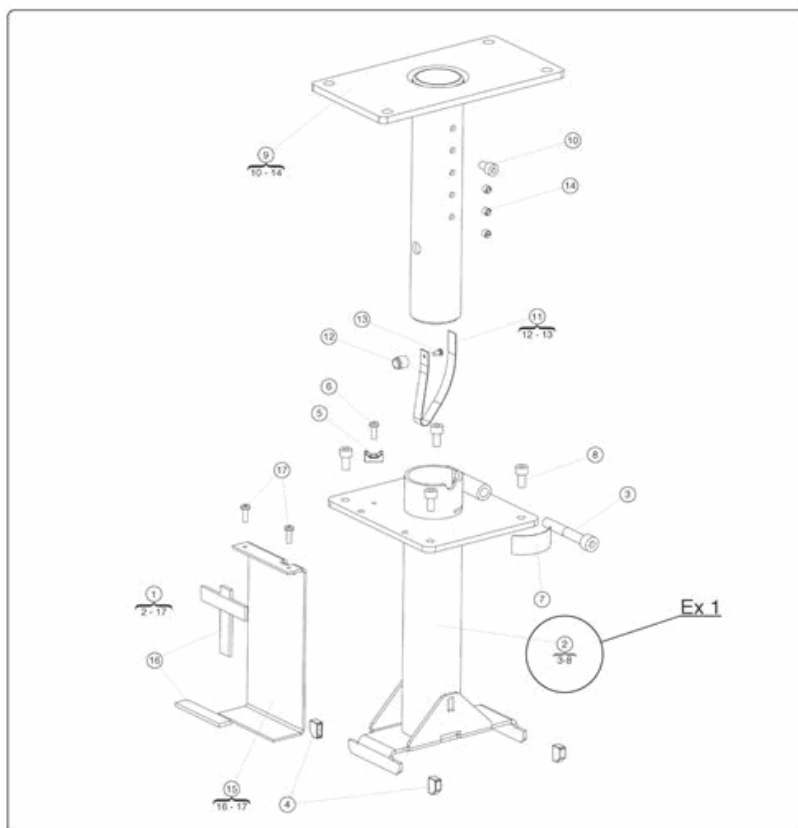**Ex 1:**

I fast sitsrör Pos. 2, ingår poserna 3till 8 som står i klamret under. (se ovan)

Ex 2:

Beställs ett Ytterrör, pos. 2, så står det att antalet är 1. Det finns med andra ord bara ett yterrör på chassit. Men om man tittar på antalet i ingående detaljer, så kan antalet variera, i tex. pos 4 är det 4 st för att yterröret ska vara komplett.



PAB no: 1261-1