

TS

Seating system for Permobil power wheelchair





Spare Part Catalog Reservdelskatalog

CONTACT PERMOBIL:

Permobil AB

Box 120
861 23 Timrå
Sverige
Tel: 060-59 59 00
Fax: 060-57 52 50
E-post: info@permobil.se

Permobil Inc. USA

6961 Eastgate Blvd.
Lebanon, TN 37090
USA
Phone: 800-736-0925
Fax: 800-231-3256
Email: info@permobilusa.com

Permobil AS

Hvamsvingen 22
2013 Skjetten
Norge
Tlf. 48 12 90 00
Fax. 63 84 48 50
E-post: firmapost@permobil.no

Permobil Europe BV

De Doom 22
NL-6419 CX Heerlen
Netherlands
Tel: +31 (0)45 564 54 90
Fax: +31 (0) 45 564 54 91
Email: europe@permobil.nl

Produced and published by Permobil AB, Sweden
PAB: 1256

Contents

Seat frame

Seat plate

Backrest frame

Backrest link manual

Backrest link power

Headrest

Armrest

Parallel motion armrest

Legrest manual

Legrest swing away, upper part

Legrest swing away, footsupport

Legrest power

Electronics

Fabrics and cushions

Ordering Information, english

Ordering Information, swedish

Notes

Innehåll

Sitsram

Ryggstödsram

Ryggstödsled, manuell

Ryggstödsled, elreglerad

Nackstöd

Armstöd

Parallellgång armstöd

Benstöd manuell

Benstöd swing away, övre delen

Benstöd swing away, fotplattor

Benstöd el


Elektronik

Tyg och dynor


Beställningsinformation, engelsk

Beställningsinformation, svensk


Noteringar

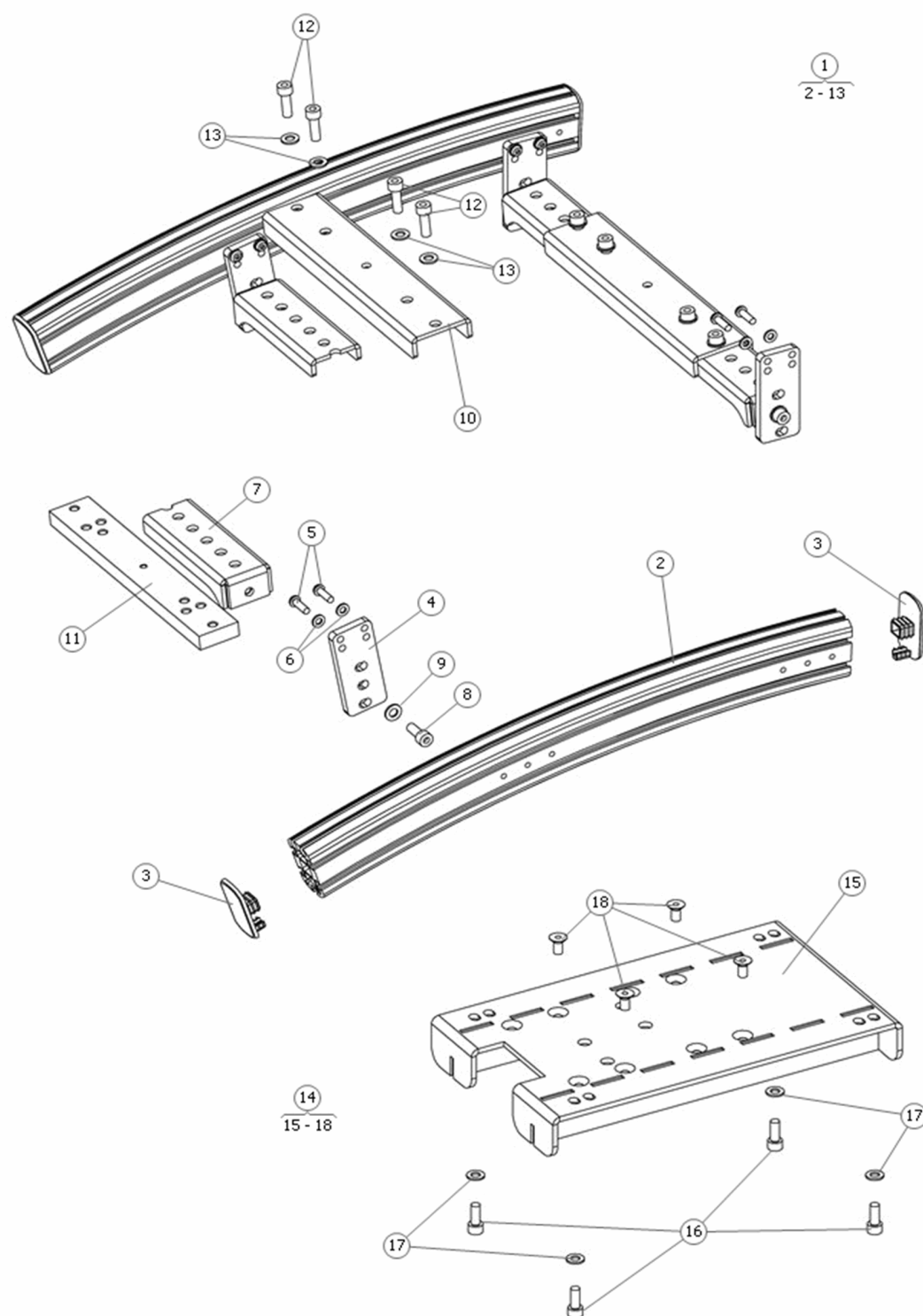

Pos	Qty	Stock Code	Description	Remark
1	1	1823990	SEAT FRAME	
2	2	1823991	PROFILE L=550	
3	4	1823992	PLASTIC PLUG	
4	4	317335	BRACKET	
5	8	601370-99-0	BOLT K6S 6x20 fzb *	
6	8	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
7	4	317614	BRACKET	
8	4	600203-99-0	BOLT MC6S 8x 20 fzb *	
9	4	600239-99-0	WASH ISO 7089 8.4x16x1.6 Fe/Zn	
10	2	317334	BRACKET	
11	2	1824750	BRACKET SEAT WIDTH	
12	8	600499-99-0	BOLT MC6S 8x 25 fzb *	
13	8	600239-99-0	WASH ISO 7089 8.4x16x1.6 Fe/Zn	
14	1	1823993	SEATPLATE INCL. SCREW	
15	1	317548	SEATPLATE	
16	4	600203-99-0	BOLT MC6S 8x 20 fzb *	
17	4	600239-99-0	WASH ISO 7089 8.4x16x1.6 Fe/Zn	
18	4	601716-99-0	SCREW ISO 10642 M8x16 Fe/Zn	

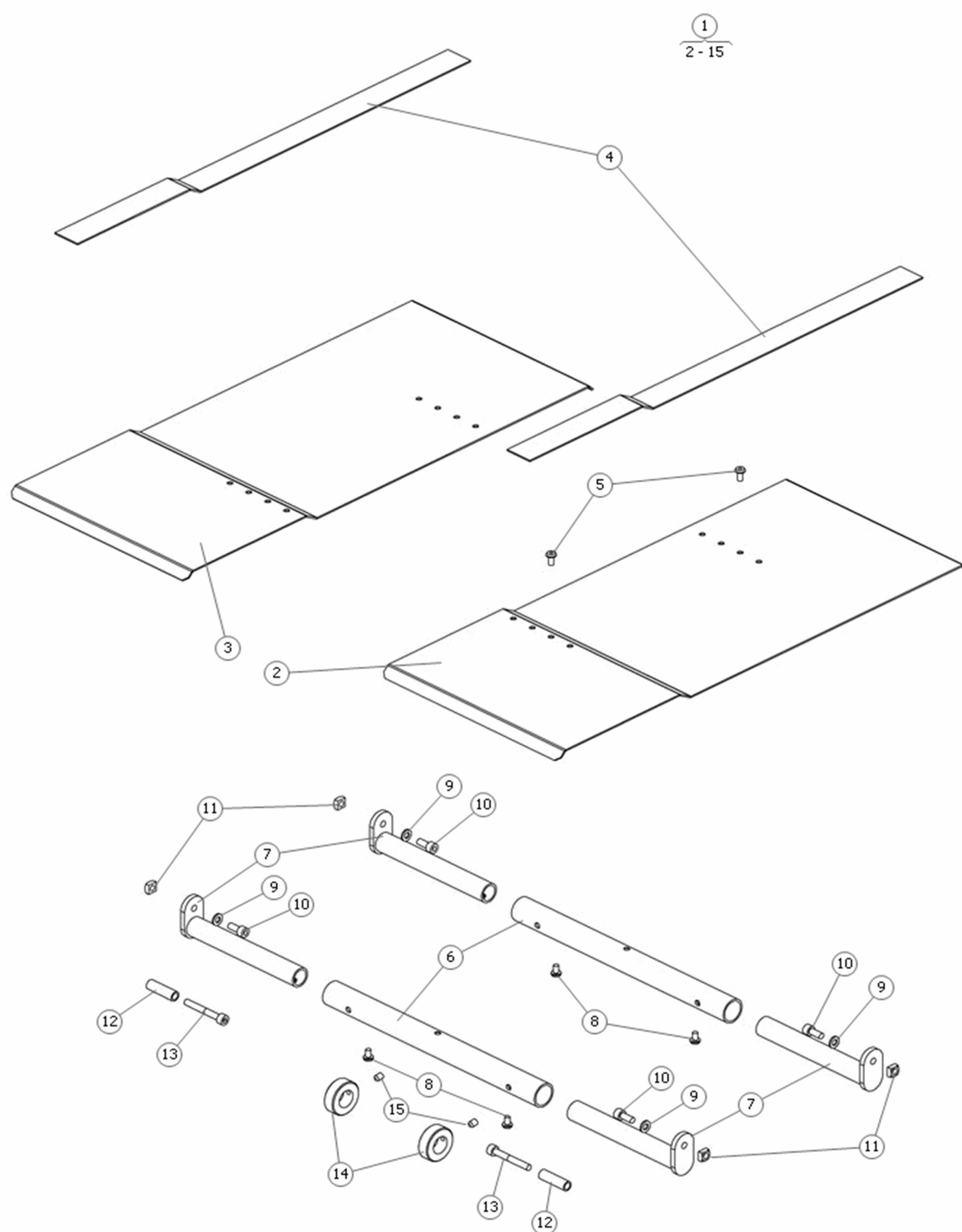


Pos	Qty	Stock Code	Description	Remark
1	1	1825046	SEATPLATE KIT	
2	1	318427	SEATPLATE LEFT	
3	1	318426	SEATPLATE RIGHT	
4	2	318442	VELCRO TAPE L=520	
5	2	601353-99-0	BOLT K6S 5x10 fzb *	
6	2	318343	TUBE	
7	4	318342	HOLDER	
8	2	600740-99-0	BOLT K6S 5x 8 fzb *	
9	4	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
10	4	600193-99-0	BOLT MC6S 6x14 fzb *	
11	4	1823929	SQUARE NUT DIN 557 M6 Fe/Zn	
12	2	318347	TUBE	
13	2	600197-99-0	BOLT MC6S 6x45 fzb *	
14	2	318811	SET COLLOR Ø35x18,2x12	
15	2	1825583	SCREW ISO 4029 M6x8 45H	

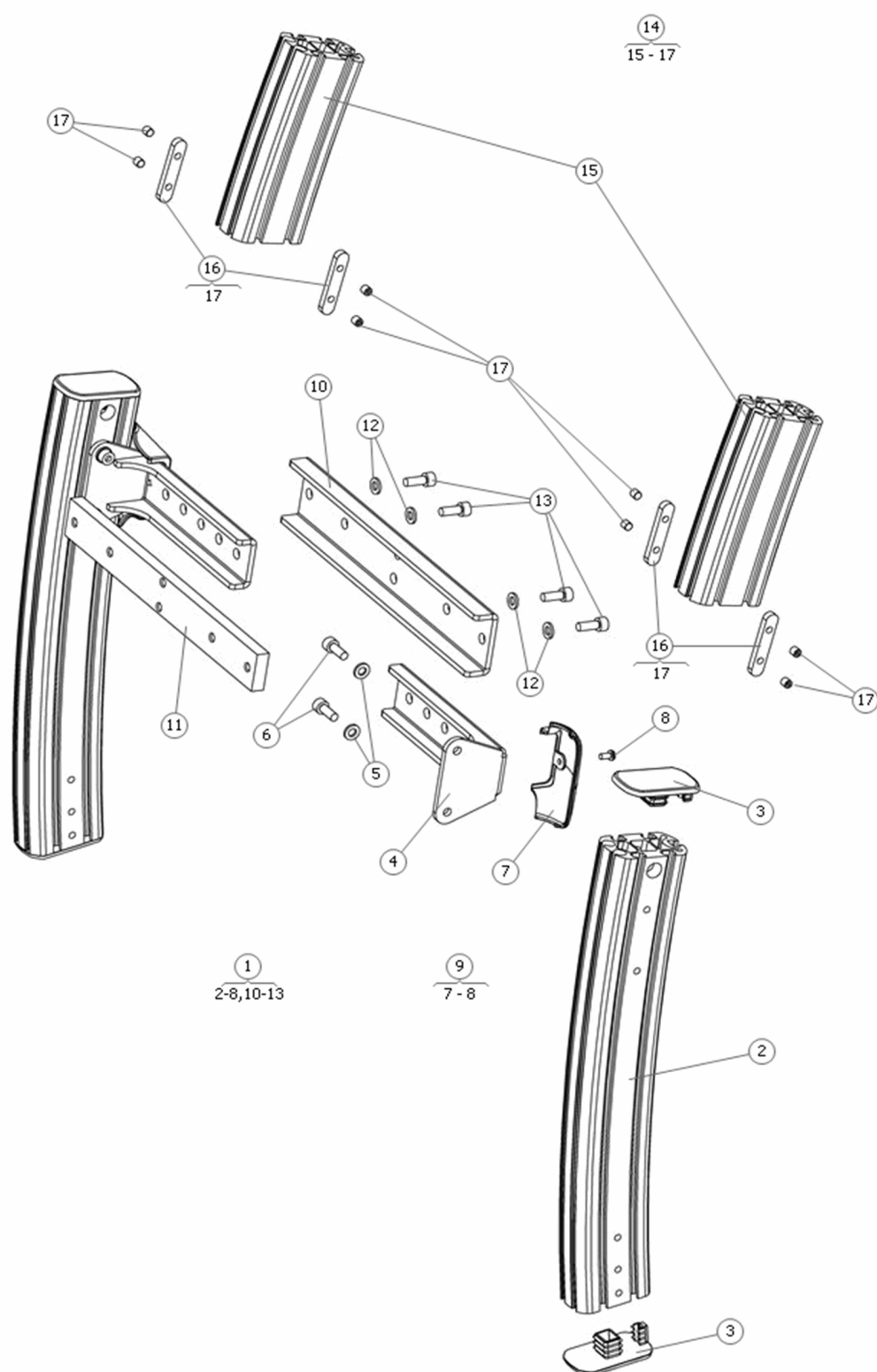


Spare Part TS

Backrest frame


Pos	Qty	Stock Code	Description	Remark
1	1	1823994	FRAME	
2	1	1823995	PROFILE L=370	
3	1	1823992	PLASTIC PLUG	
4	1	317427	BRACKET	
5	1	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
6	1	600743-99-0	BOLT MC6S 6x16 fzb *	
7	1	1824716	COVER	
8	1	601784-00-0	BOLT K6S 4x 8 BLACK *	
9	1	1823996	COVER	
10	1	317429	BRACKET	
11	1	1824751	BRACKET BACKREST WIDTH	
12	1	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
13	1	600194-99-0	BOLT MC6S 6x20 fzb *	
14	1	1823997	BACKREST EXTENSION KIT	
15	1	317453	PROFILE L=150	
16	1	1823998	NUT SLIDE BAR 50mm.	Incl. two screwas (item 600214)
17	1	600214	SCREW ISO 4029 M6x6 45H	

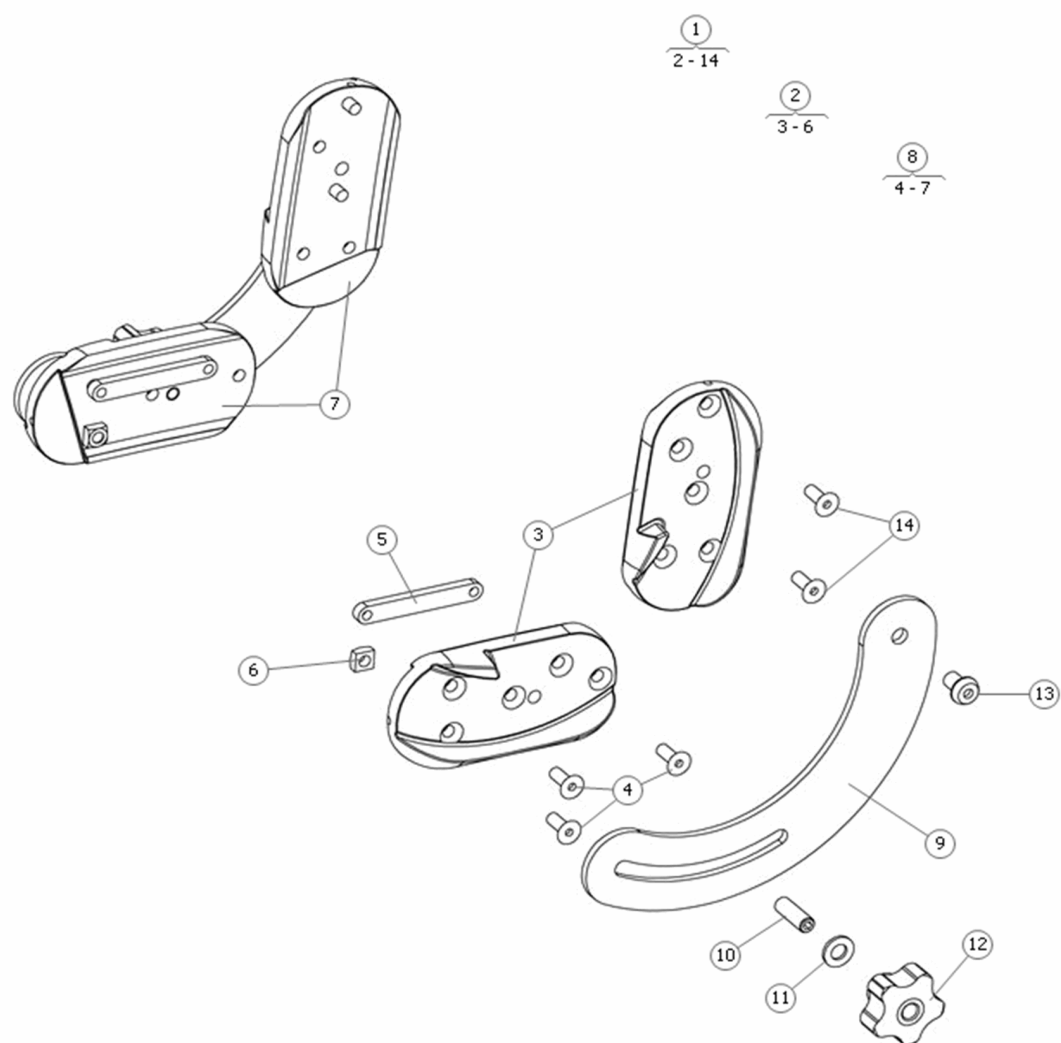


Pos	Qty	Stock Code	Description	Remark
1	1	1823931	BACKREST HINGE MANUAL	
2	1	1823926	BRACKET KIT LEFT	
3	2	1824712	BACKREST LEFT	
4	6	600541-99-0	BOLT MF6S 6x16 fzb *	
5	2	1823930	SLIDE BAR M6 C-C 58mm	
6	2	1823929	SQUARE NUT DIN 557 M6 Fe/Zn	
7	2	1824713	BACKREST RIGHT	
8	1	1823927	BRACKET KIT RIGHT	
9	2	317289	PLATE BOW	
10	2	613215	SCREW ISO 4026 M8x25 45H	
11	2	600239-99-0	WASH ISO 7089 8.4x16x1.6 Fe/Zn	
12	2	1823928	LOCK HANDLE M8	
13	2	603275	SCREW ISO 7380 M8x12 Fe/Zn	
14	4	600541-99-0	BOLT MF6S 6x16 fzb *	

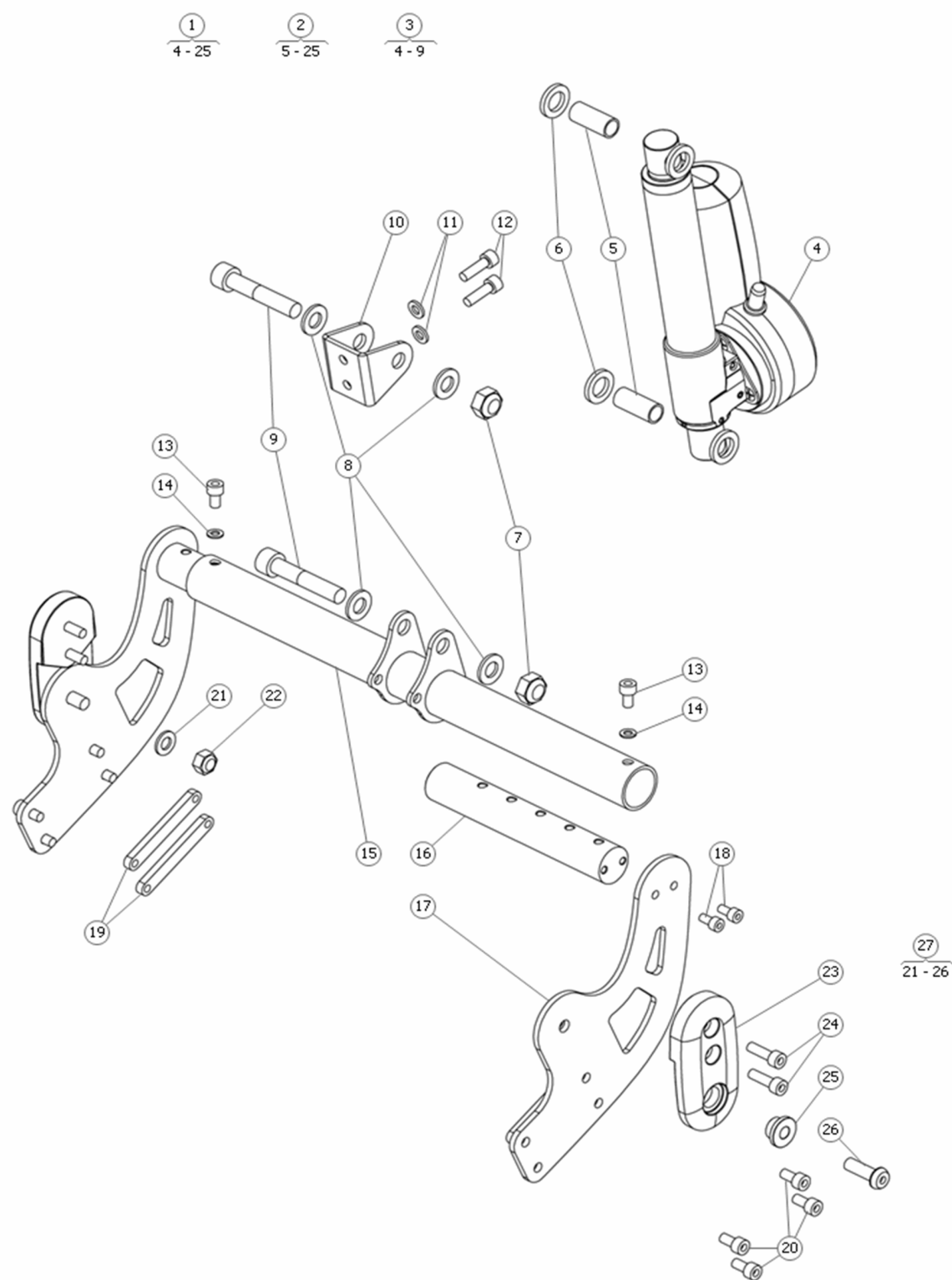


Pos	Qty	Stock Code	Description	Remark
1	1	1823934	BACKREST HINGE POWER	
2	1	1824714	BACKREST HINGE	
3	1	1825287	ACTU BACK PWR TS REAC	
4	1	1825279	ACTU RE25 156+70 (stroke) 2x13,1	
5	1	302330-99-0	ACTU SLEEVE 30	
6	1	1825286	WASHER	
7	1	600229-99-0	LOCK NUT M10 FZB *	
8	1	600241-99-0	WASH BRB 10.5x20x2 fzb *	
9	1	601231-99-0	BOLT MC6S 10x 55 fzb *	
10	1	317517	BRACKET	
11	1	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
12	1	600194-99-0	BOLT MC6S 6x20 fzb	
13	1	600888-99-0	BOLT MC6S 6x10 fzb *	
14	1	600237-99-0	WASH BRB 5.3x10x 1 fzb *	
15	1	317515	BRACKET	
16	1	317509	SHAFT	
17	1	317482	HINGE	
18	1	600189-99-0	BOLT MC6S 5x10 fzb *	
19	1	1823930	NUT SLIDE BAR M6 58mm. LONG	
20	1	600192-99-0	BOLT MC6S 6x12 fzb *	
21	1	600239-99-0	WASH ISO 7089 8.4x16x1.6 Fe/Zn	
22	1	600228-99-0	LOCK NUT M8 FZB *	
23	1	1823933	BACKR HINGE BRACKET PWR TS	
24	1	600194-99-0	BOLT MC6S 6x20 fzb	
25	1	1823932	BEARING	
26	1	602165-99-0	BOLT K6S 8x 25 FZB	
27	1	1823946	BOLT KIT BACKR PWR TS	

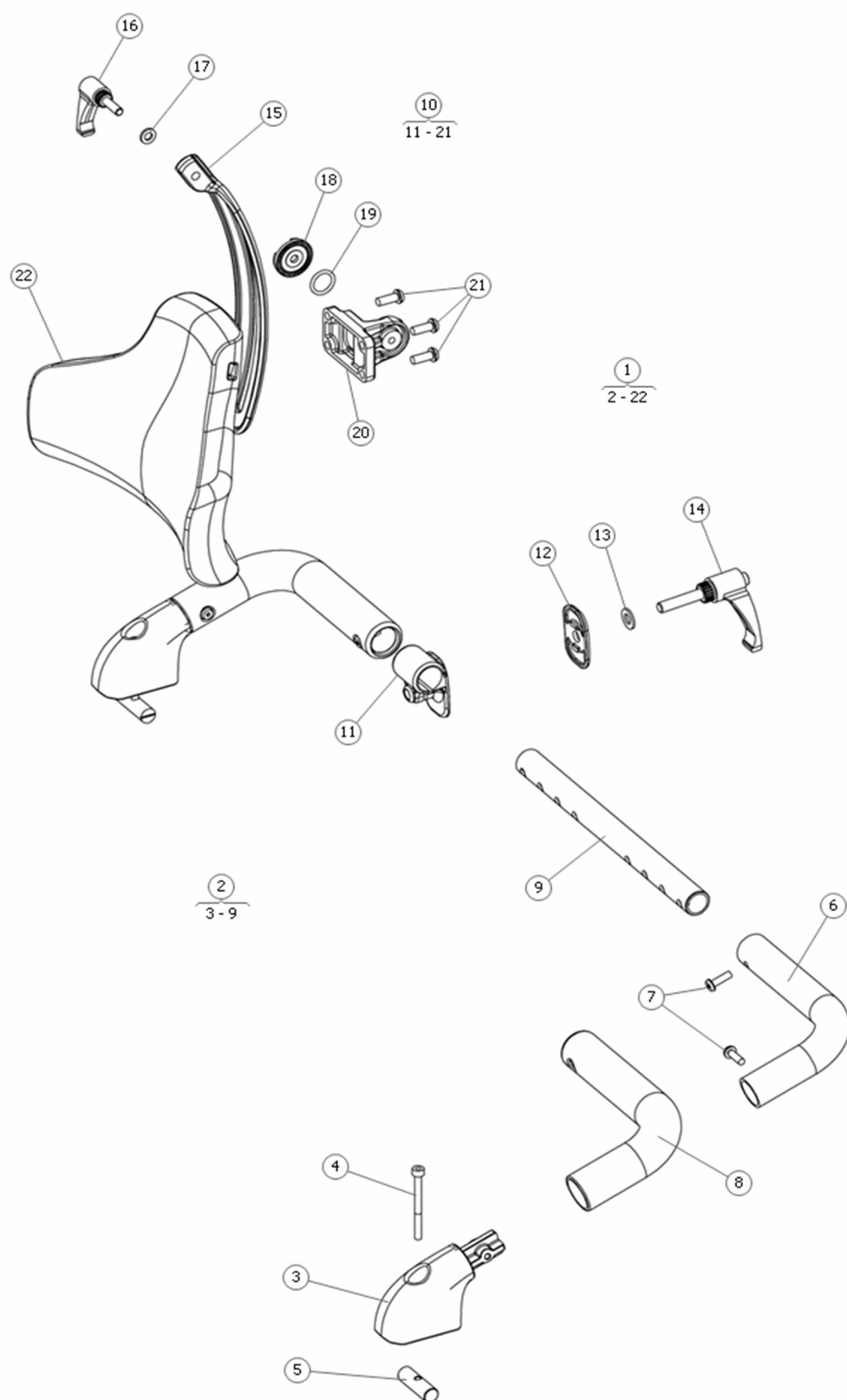


Pos	Qty	Stock Code	Description	Remark
1	1	1824000	HEAD SUPPORT	
2	1	1823999	PUSH HANDLES	
3	1	1824001	BRACKET PUSH HANDLES	BRACKET, SCREWS AND NUTS
4	1	317793-99-0	SCREW GLUE DRAWING TS	
4	1	602495	SCREW ISO 4762 M5x55 Fe/Zn	
5	1	317523-99-0	NUT PIN Ø12, L=38, M5	
6	1	317539	TUBE	
7	1	601170-99-0	BOLT K6S 5x16 fzb *	
8	1	317524	PUSH HANDLES	
9	1	317525	TUBE	
10	1	1823234	HEAD SUPPORT BRACKET	
11	1	318116	LOCK ADJUSTMENT	
12	1	318118	WASHER	
13	1	600239-99-0	WASH ISO 7089 8.4x16x1.6 Fe/Zn	
14	1	1822870	ADJUSTABLE HANDLE M8x35	
15	1	318117	LINK ARM	
16	1	1822869	ADJUSTABLE HANDLE M6x20	
17	1	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
18	1	1823502	ADJUSTMENT DISC (TEETH) UNITRAC	
19	1	1823501	RUBBER RING Ø15,3x2,4 NBR 70	
20	1	1821887	BRACKET HEADSUPPORT UNITRAC	
21	1	601675-99-0	BOLT K6S 6x16 fzb *	
22	1	1822868	HEAD SUPPORT	

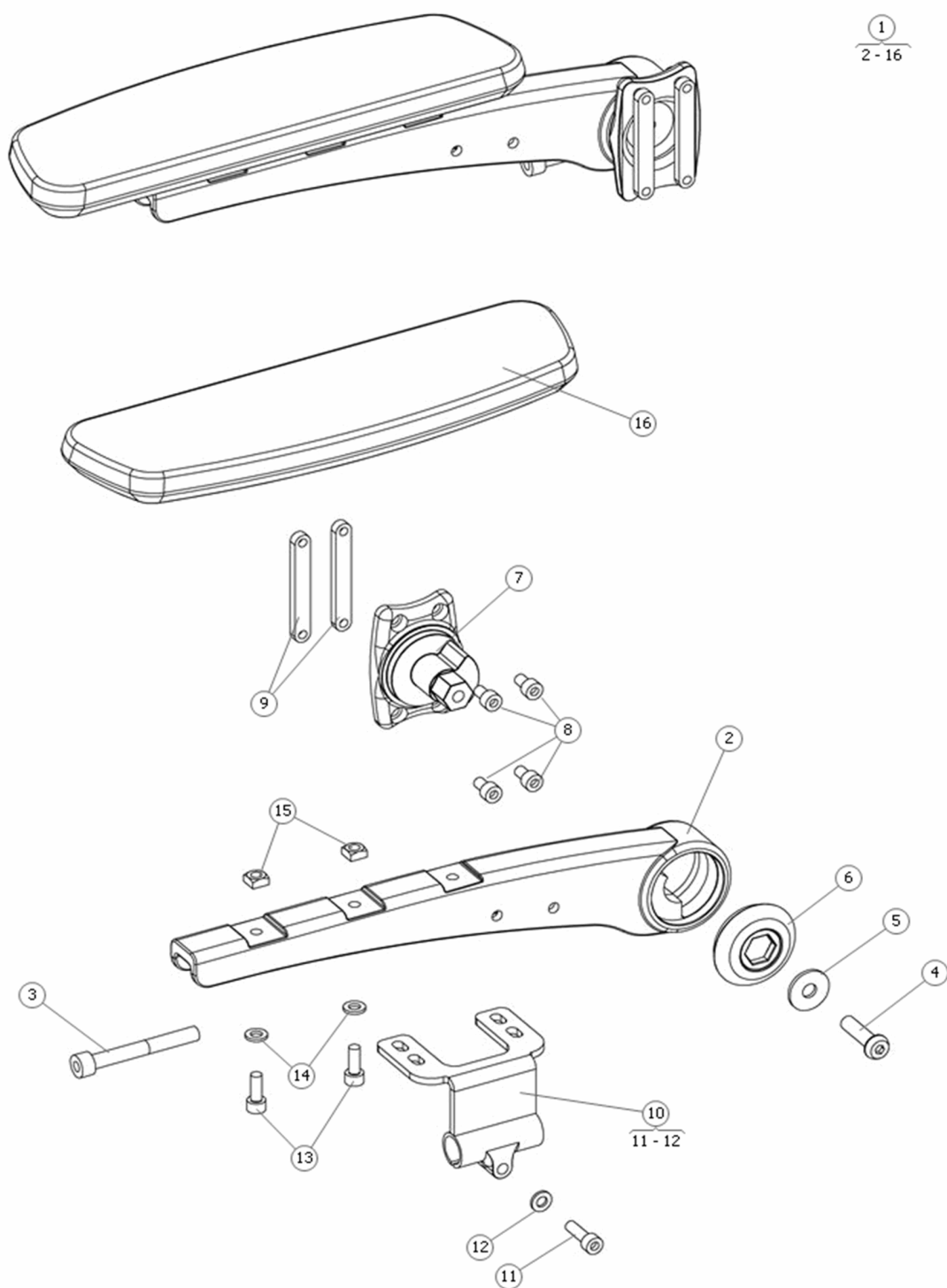


Pos	Qty	Stock Code	Description	Remark
1	1	1823989	ARMREST ASSY	
2	1	317478	ARMREST	
3	1	612997	BOLT ISO 4762 8x 50 fzb *	
4	1	603275	SCREW ISO 7380 M8x12 Fe/Zn	
5	1	612393-99-0	WASHER ISO 7093-1 Fe/Zn 8,4x24x2	
6	1	315842	SOCKET	
7	1	317438	BRACKET	
8	1	600888-99-0	BOLT MC6S 6x10 fzb *	
9	1	1823930	SLIDE BAR M6 C-C 58mm	
10	1	1824148	PANELHOLDER	
11	1	600194-99-0	BOLT MC6S 6x20 fzb	
12	1	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
13	1	600743-99-0	BOLT MC6S 6x16 fzb *	
14	1	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
15	1	1823929	SQUARE NUT DIN 557 M6 Fe/Zn	
16	1	1821696	SUPPORT	

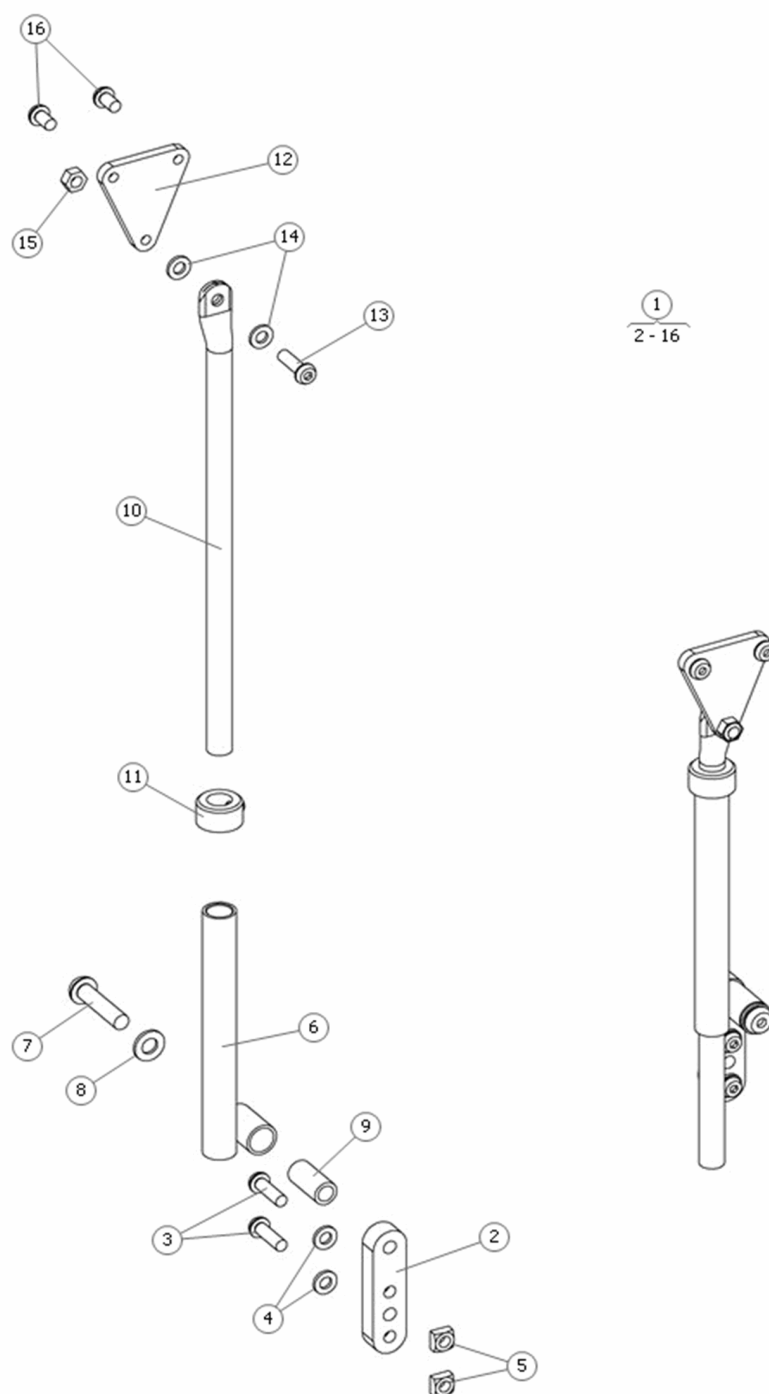


Pos	Qty	Stock Code	Description	Remark
1	1	1823988	PARALLEL MOTION KIT	
2	2	317480	BAR	
3	4	601370-99-0	BOLT K6S 6x20 fzb *	
4	4	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
5	4	1823929	SQUARE NUT DIN 557 M6 Fe/Zn	
6	2	317831	TUBE	
7	2	602373	SCREW ISO 7380 M8x35 Fe/Zn	
8	2	600239-99-0	WASH ISO 7089 8.4x16x1.6 Fe/Zn	
9	2	317832	SOCKET 8,3x11,3x25	
10	2	317798	TUBE	
11	2	602322-99-0	SET COLLAR DIN 705 A12 Fe/Zn Ø12xØ22x12	
12	2	317481	HOLDER	
13	2	601370-99-0	BOLT K6S 6x20 fzb *	
14	4	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
15	2	600227-99-0	LOCK NUT M6 FZB *	
16	4	600736-99-0	BOLT K6S 6x10 fzb *	

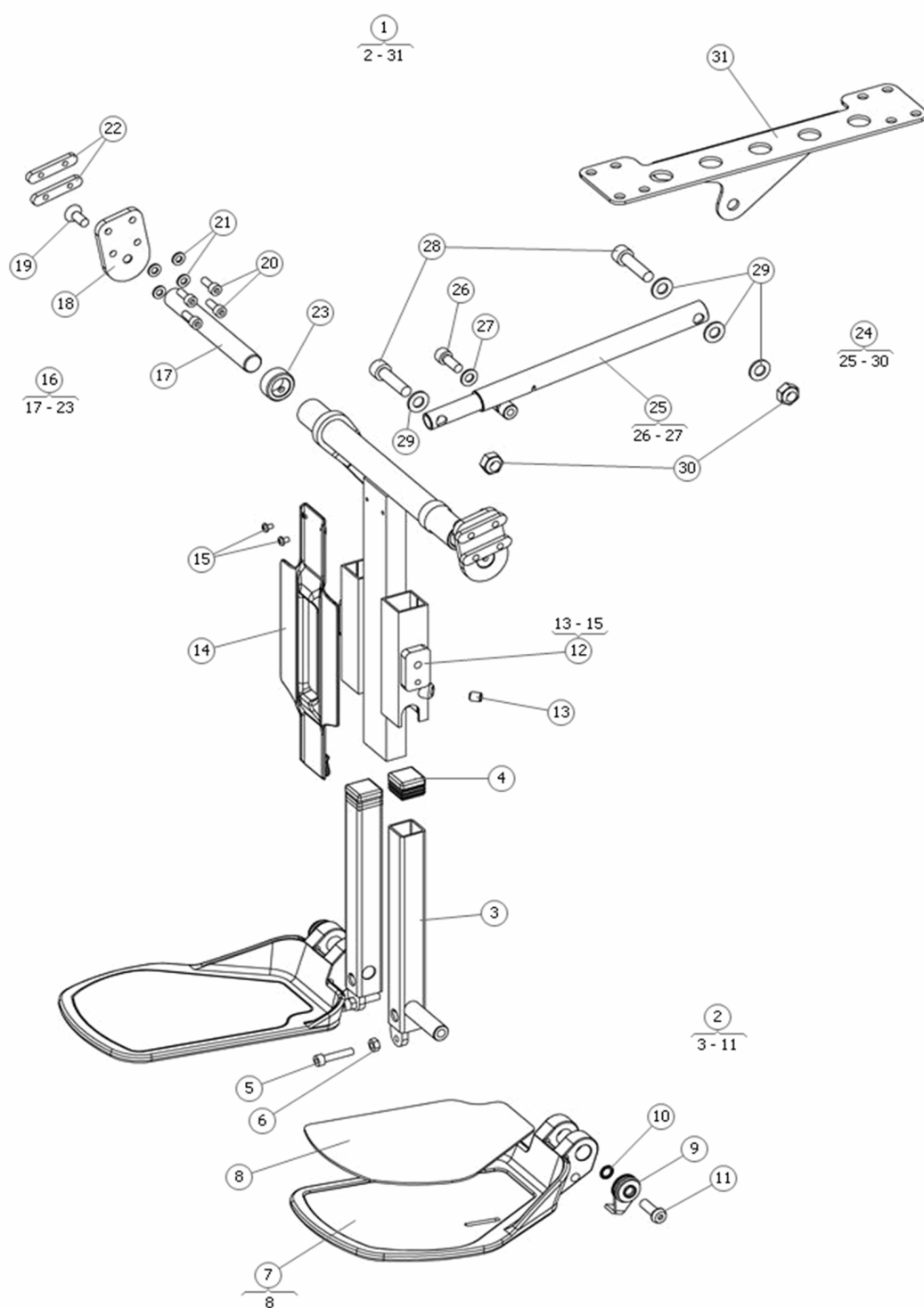


Pos	Qty	Stock Code	Description	Remark
1	1	1824112	LEGREST	
2	1	1824151	FOOTPLATE INCL. BRACKET LEFT	BLACK
2	1	1824152	FOOTPLATE INCL. BRACKET RIGHT	BLACK
3	1	312912	BRACKET FOOTPLATE RIGHT	INCL. FRICTION PLATE AND SCREWS
3	1	312913	BRACKET FOOTPLATE LEFT	INCL. FRICTION PLATE AND SCREWS
4	1	601236-00-0	PLASTIC PLUG 25x25 (21-22x21-22)	
5	1	600196-99-0	BOLT MC6S 6x30 fzb *	
6	1	600232-99-0	NUT M6M 6 fzb *	
7	1	1827562	FOOTPL DUAL V.2	
7	1	1827563	FOOTPL DUAL V.2	
8	1	1820714	SLIP PROTECT SET	NARROW
8	1	1820716	SLIP PROTECT SET	WIDE
9	2	308371-99-0	FRICTIONBRAKE FOOTPLATE	POS. 9,10,11
10	1	602127-99-0	WASH SCHNORR 8,4*13*0,8 *	
11	1	602165-99-0	BOLT K6S 8x 25 FZB	
12	1	1824114	LEGREST	NO COVER AND SCREWS
13	2	600737-99-0	SET SCREW SK6SS 8x10 *	
14	1	1825199	COVER LEGREST	
15	2	602243	SCREW ISO 7045 M4x6 Fe/Zn	
16	2	1824115	CONSOLE	
17	1	317290	SHAFT	
18	1	317288	PLATE	
19	1	600764-99-0	BOLT MF6S 8x20 fzb *	
20	4	600743-99-0	BOLT MC6S 6x16 fzb *	
21	4	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
22	4	1823998	NUT SLIDE BAR M6 50mm, SHORT	
23	1	307421	SET COLLOR Ø28xØ18,2x12	
24	1	000000	MANUAL ADJUST. KIT	NOT AVAILABLE
25	1	304484-40-0	ADJ LOCK TUBE 225-380	
26	1	601127-99-0	BOLT MC6S 8x 22 fzb *	
27	1	600239-99-0	WASH ISO 7089 8.4x16x1.6 Fe/Zn	
28	2	601367-99-0	BOLT MC6S 10x 40 fzb *	
29	4	600241-99-0	WASH BRB 10.5x20x2 fzb *	
30	2	600229-99-0	LOCK NUT M10 FZB *	
31	1	1824149	ACTU BRACKET LEGREST TS	

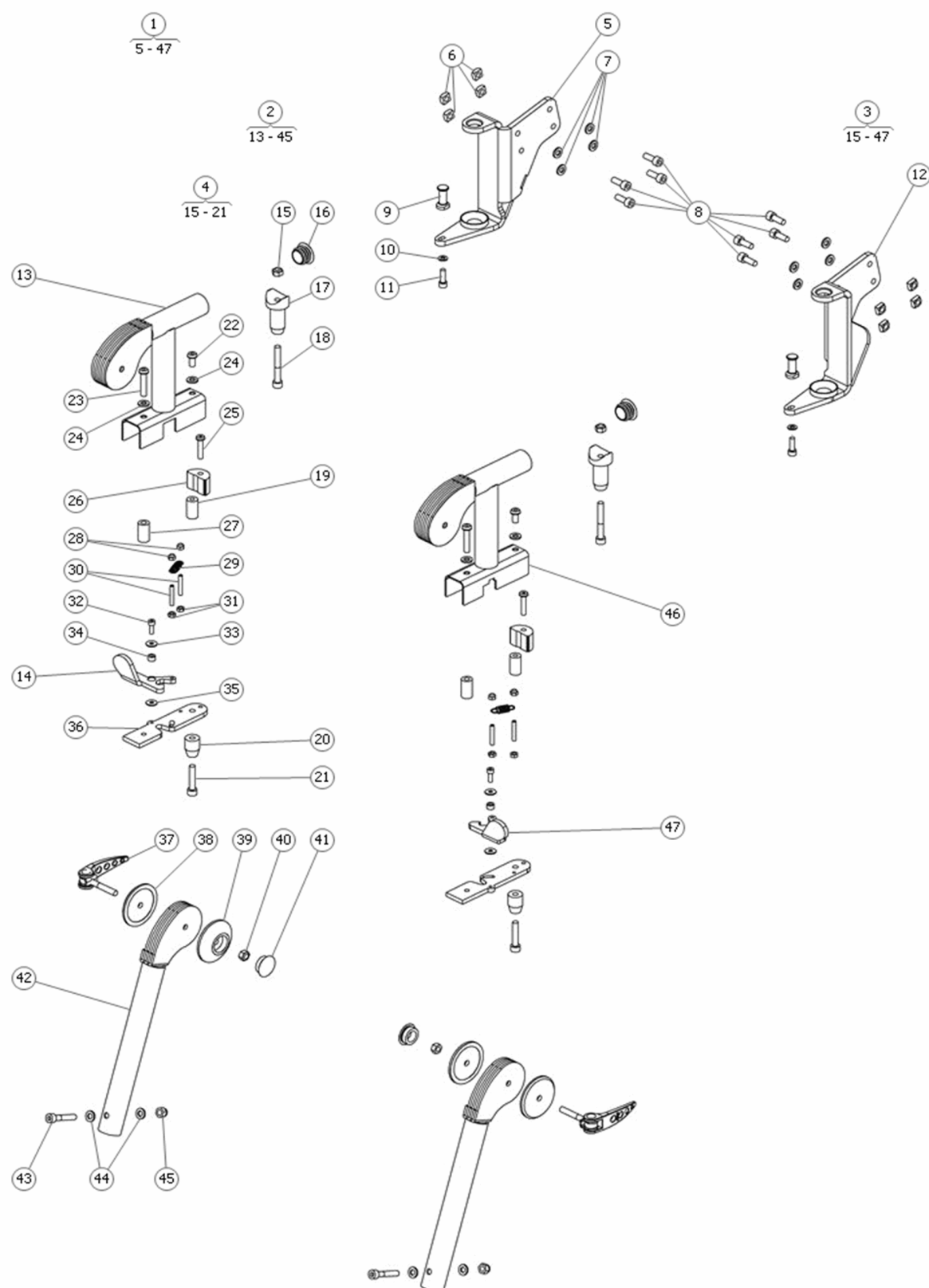

Pos	Qty	Stock Code	Description	Remark
1	1	1825076	LEGREST, UPPER	
2	1	317553	HOLDER, RIGHT	
3	1	317552	HOLDER, LEFT	
4	1	1821147	PIVOT BUSHING KIT	
5	1	317295	CONSOLE	
6	4	1823929	NUT, DIN 557 M6 fzb	
7	4	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
8	4	600743-99-0	BOLT MC6S 6x16 fzb *	
9	1	313475	COTTER	
10	1	600237-99-0	WASH BRB 5.3x10x 1 fzb *	
11	1	601165-99-0	BOLT MC6S 5x16 fzb *	
12	1	317296	CONSOLE	
13	1	317316	HOLDER	
14	1	316647	CATCH	
15	1	600227-99-0	LOCK NUT M6 FZB *	
16	1	313497	PLASTIC PLUG	
17	1	313467	HUB SOCKET	
18	1	600197-99-0	BOLT MC6S 6x45 fzb *	
19	1	313463	SPACER	
20	1	313466	HUB SOCKET	
21	1	600196-99-0	BOLT MC6S 6x30 fzb *	
22	1	601422-99-0	BOLT K6S 6x12 fzb *	
23	1	602035	SCREW, ISO 7380 M6x30 fzb	
24	2	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
25	1	600754-99-0	BOLT K6S 5x25 fzb *	
26	1	313461	SPACER	
27	1	313465	SPACER	
28	2	600225-99-0	LOCK NUT M4 FZB *	
29	1	612706	SPRING	
30	2	601559	SCREW, ISO 4026 M4x25	
31	2	600231-99-0	NUT M6M 4 fzb *	
32	1	600184-99-0	BOLT MC6S 4x12 fzb *	
33	1	600829-99-0	WASH BRB 4.3x12x 1 fzb DIN 9021 B	
34	1	313477	BUSH	
35	1	600829-99-0	WASH BRB 4.3x12x 1 fzb DIN 9021 B	
36	1	313458	PLATE	
37	1	613746	LOCK HANDLE	
38	1	306896-00-0	PRESSURE PLATE OUTER ADJ LEGREST MPS	
39	1	306967-00-0	PRESSURE PLATE INNER ADJ LEGREST MPS	
40	1	600227-99-0	LOCK NUT M6 FZB *	
41	1	601656-00-0	PLAST PLUG 25-2025 H.LIGHT/LEGR M90 M90T/MPS	
42	1	317314	HINGE	
43	1	601333-99-0	BOLT MC6S 6x35 fzb	
44	2	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
45	1	1821261	NUT DIN 982 M6 fzb	
46	1	317312	HOLDER	
47	1	316648	CATCH	

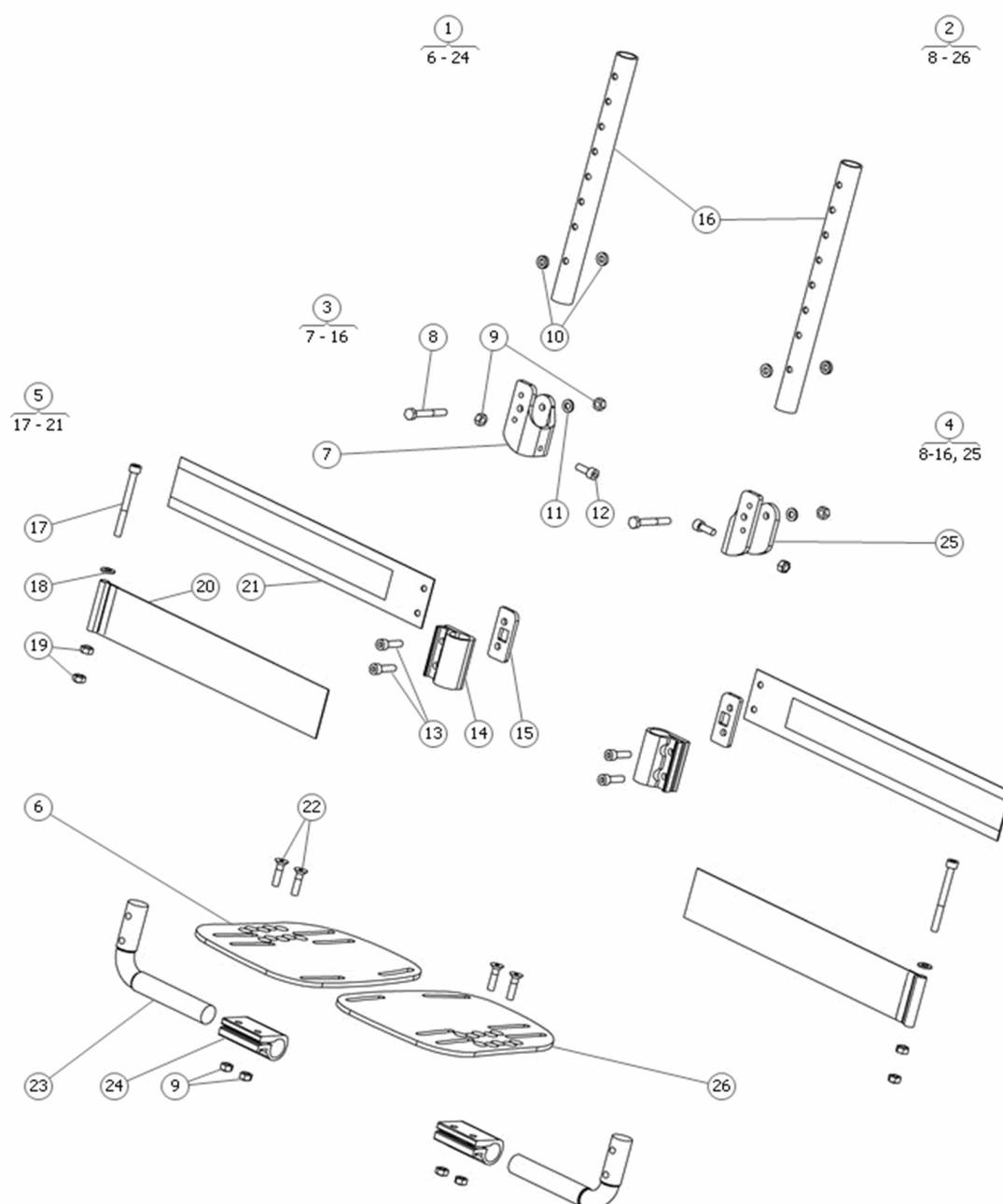


Pos	Qty	Stock Code	Description	Remark
1	1	1823215	FOOTSUPPORT	
2	1	1823216	FOOTSUPPORT	
3	1	1823220	FOOTPL CLAMP/HINGE RIGHT	
4	1	1823219	FOOTPL CLAMP/HINGE LEFT	
5	1	1823221	FOOTSTRAPS	
6	1	315687	FOOTPLATE RIGHT	
7	1	315362	BRACKET	
8	1	613367	BOLT, ISO 4014 M6x45 fzb	
9	2	600227-99-0	LOCK NUT M6 FZB *	
10	2	601246-99-0	WASH TBR SUB 6.4x12x3 fzb	
11	1	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
12	2	600743-99-0	BOLT MC6S 6x16 fzb *	
13	2	600692-99-0	BOLT MC6S 6x22 fzb *	
14	1	315364	CLAMP	
15	1	315792	SPACER	
16	1	315361	TUBE	
17	1	602039-99-0	SCREW, ISO 4762 M6x65 fzb	
18	1	316881	WASHER	
19	2	600227-99-0	LOCK NUT M6 FZB *	
20	1	315923	WEB BELT	
21	1	315971	WEB BELT	
22	2	600735	SCREW, ISO 10642 M6x25 fzb	
23	1	315365	BAR	
24	1	315364	CLAMP	
25	1	315363	BRACKET	
26	1	315688	FOOTPLATE, LEFT	

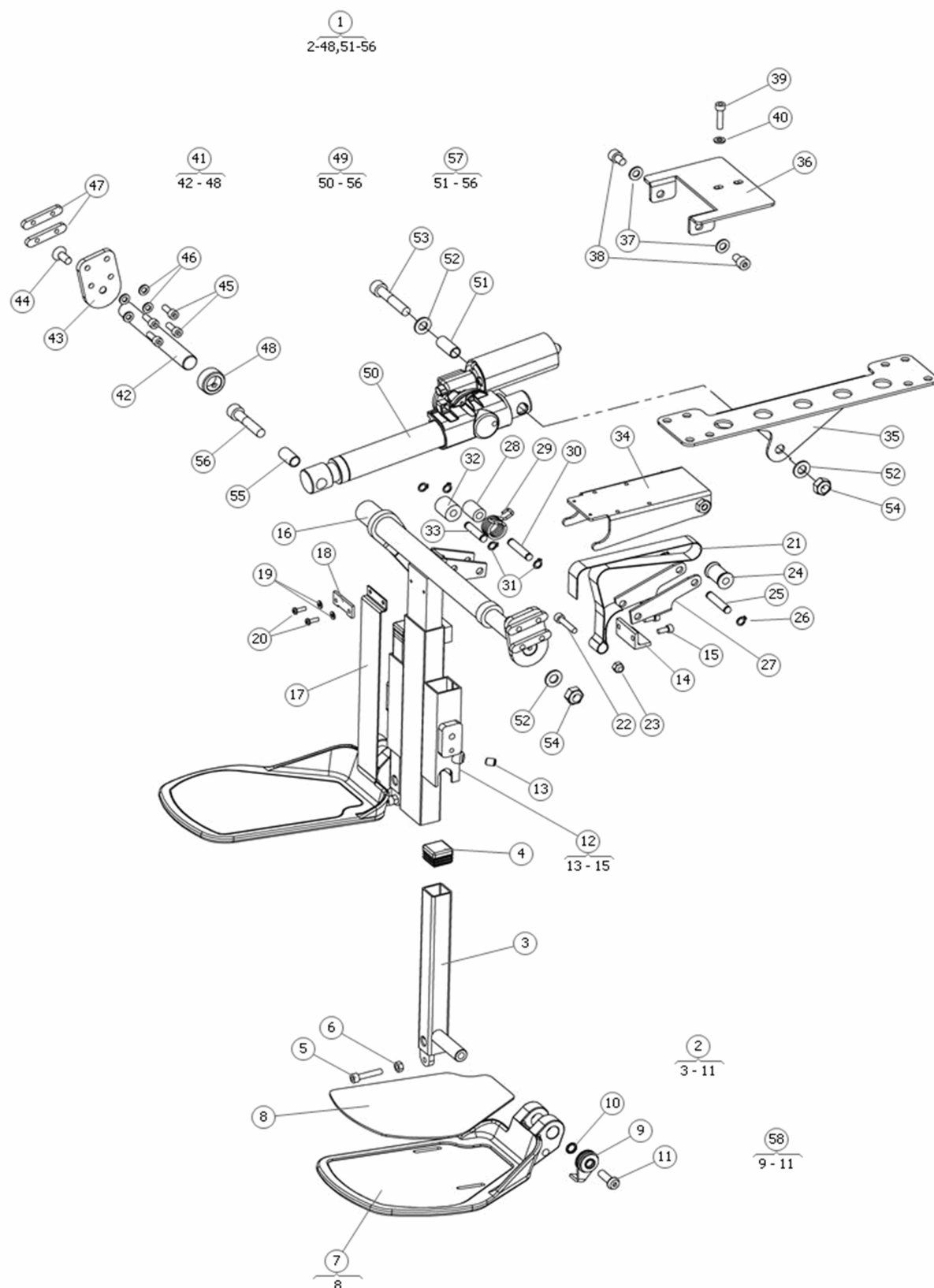


Pos	Qty	Stock Code	Description	Remark
1	1	1824111	LEGREST	
2	1	1824151	FOOTPLATE INCL. BRACKET LEFT	BLACK
2	1	1824152	FOOTPLATE INCL. BRACKET RIGHT	BLACK
3	1	312912	BRACKET FOOTPLATE RIGHT	
3	1	312913	BRACKET FOOTPLATE LEFT	
4	1	601236-00-0	PLAST PLUG 56-25-25	
5	1	600196-99-0	BOLT MC6S 6x30 fzb *	
6	1	600232-99-0	NUT M6M 6 fzb *	
7	1	1827562	FOOTPL DUAL V.2	Right, incl bracket. Black.
7	1	1827563	FOOTPL DUAL V.2	Left, incl bracket. Black.
8	1	1820714	RUBBER PROTECT, PAIR	LEFT AND RIGHT
8	1	313957	RUBBER PROTECT RIGHT	
8	1	313968	RUBBER PROTECT LEFT	
9	1	306983	FRICTIONS BRAKE	
10	1	602127-99-0	WASH SCHNORR 8,4*13*0,8 *	
11	1	602165-99-0	BOLT K6S 8x 25 FZB	
12	1	305240-28-0	OUT SECT FOOTPL BRACKET CII	
12	1	305240-38-0	OUT SECT FOOTPL BRACKET CII	
13	2	600737-99-0	SET SCREW SK6SS 8x10 *	
14	1	305237-99-0	WEB BRACKET LEGREST CII GN	
15	2	600169-99-0	BOLT MC6S 5x12 fzb *	
16	1	1824113	INNER PROFIL	
17	1	305183-28-0	PROTECT PLATE LEGREST CII	
18	1	309132	WEB.BELT HOLDER	
19	2	601296-99-0	WASH BRB 4.3x 9x0.8 fzb STD	
20	2	601102-99-0	BOLT K6S 4x12 fzb *	
21	1	305245-00-0	WEB BELT CALF REST CII	
22	1	600196-99-0	BOLT MC6S 6x30 fzb *	
23	1	1821261	Nut DIN 982 M6 fzb	
24	1	1821259	ROLLER	
25	1	305221-99-0	SHAFT 43mm ROLL	
26	2	600503-99-0	CIRCLIP SGA 8 *	
27	1	1821089	ROLLER BRACKET	
28	1	1821258	ROLLER	
29	1	1821260	SPRING	
30	1	305221-99-0	SHAFT 43mm ROLL	
31	4	600503-99-0	CIRCLIP SGA 8 *	
32	1	305224-99-0	ROLL 23mm CII	
33	1	305223-99-0	SHAFT 36mm ROLL	
34	1	317688	SLIDE BAR	
35	1	1824149	ACTU BRACKET LEGREST TS	
36	1	1824150	LEGREST HOLDER TS	
37	2	600239-99-0	WASH ISO 7089 8.4x16x1.6 Fe/Zn	
38	2	600705-99-0	BOLT MC6S 8x 12 fzb *	
39	1	600195-99-0	BOLT MC6S 6x25 fzb *	
40	1	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
41	2	1824115	CONSOLE	
42	1	317290	SHAFT	
43	1	317288	PLATE	
44	1	600764-99-0	BOLT MF6S 8x20 fzb *	
45	4	600743-99-0	BOLT MC6S 6x16 fzb *	
46	4	600238-99-0	WASH BRB ISO 7089 6.4x12x1.6 fzb STD	
47	2	1824717	PLATE	
48	1	307421	SET COLLOR Ø28xØ18,2x12	
49	1	1825091	ACTU INCL. SCREW KIT	ACTUATOR RE25 INCL. SCREW ASSEMBLY
50	1	612421-99-0	ACTU LA28.1-100-24 MOTOR TURNED 180°	ACTU LINAK (OLD VERSION) DO NOT USE!
50	1	612804-99-0	ACTU REAC 25.4.100 M1.C33BR.OH.L(156+STROKE 100MM)	ACTU REAC (NEW VERSION)
51	1	1825219	SPACER L=24	To Linak actuator (Item 612421 or 612484)
52	3	600241-99-0	WASH BRB 10.5x20x2 fzb *	
53	1	601231-99-0	BOLT MC6S 10x 55 fzb *	To Linak actuator (Item 612421 or 612484)
54	2	600229-99-0	LOCK NUT M10 FZB *	
55	1	1825219	SPACER L=24	
56	1	612778	SCREW, ISO 7380 M10x50 10.9 fzb	To Linak actuator (Item 612421 or 612484)
57	1	1825090	INST ASSY LEGR ACTU	
58	1	308371-99-0	FRICTIONBRAKE FOOTPLATE	


If 3 or 4 actuators are used on seat.

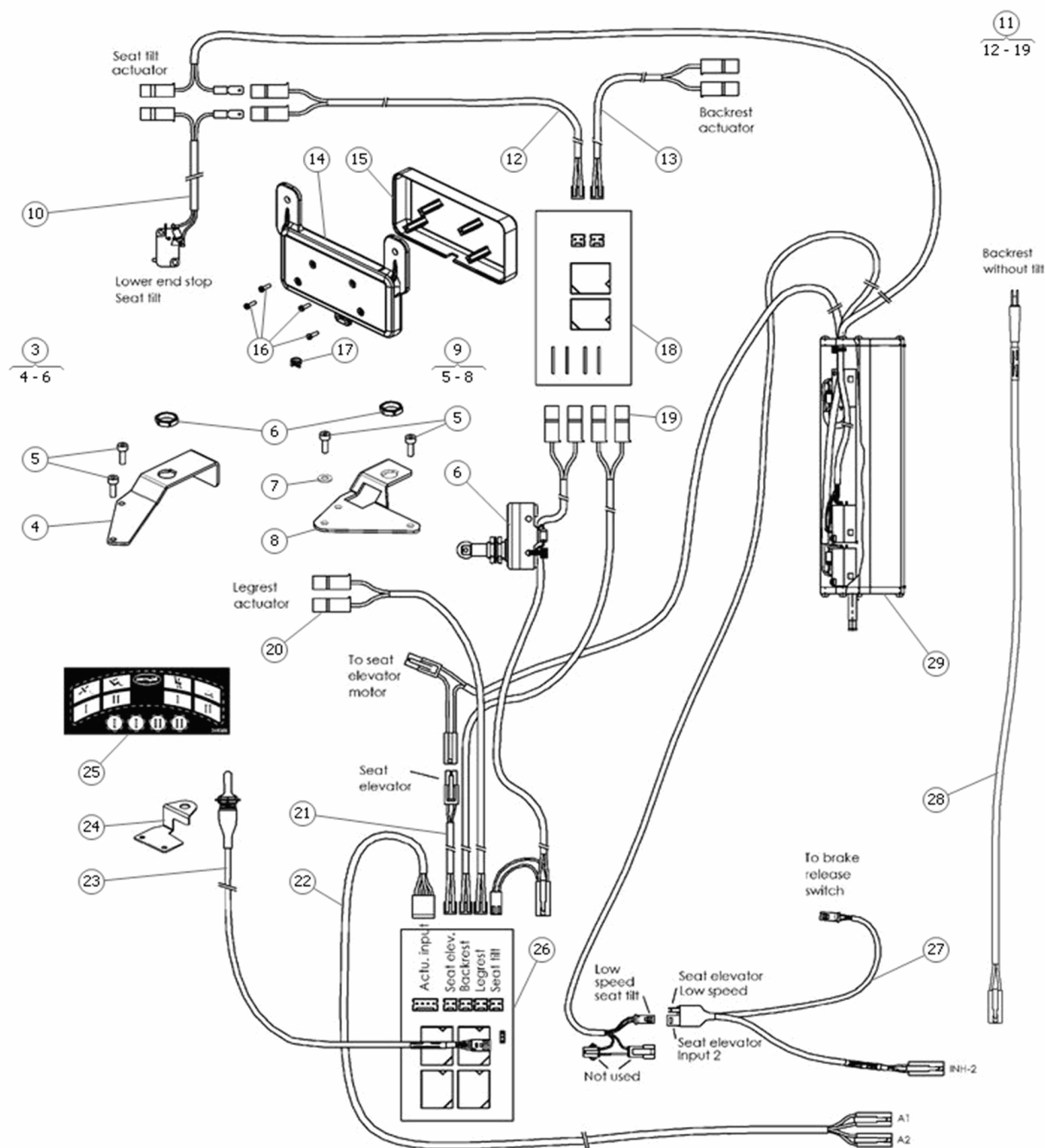

If 1 or 2 actuators are used on seat.

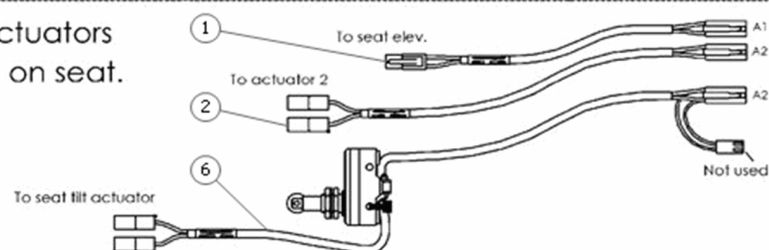

Pos	Qty	Stock Code	Description	Remark
1	1	1822364	CABLE SEAT ELEV. VR2 C300 C350 L=500mm.	
2	1	1822365	CABLE SEAT TILT VR2 C300 C350	
3	1	1825445	SWITCH	TS for C350
4	1	317179	HOLDER	
5	2	600184-99-0	BOLT MC6S 4x12 fzb *	
6	1	1824116	CABLE/SW SEAT TILT STOP TS	
7	1	600237-99-0	WASH BRB 5.3x10x 1 fzb *	
8	1	313763	HOLDER	
9	1	1825444	SWITCH	TS for C300
10	1	1824177	CABLE LOWER END STOP TILT	
11	1	1824163	ANGLE STOP TS	INCL. BOX, PCB, CABLES
12	1	306743-99-0	CABLE ACTU CONTROL *	
13	1	1824161	CABLE ACTUATOR 300mm.	
14	1	318008	BOX ANGLE SENSOR	
15	1	318010	LID ANGLE SENSOR	
16	4	613425	SCREW, 3x10 Fe/Zn	
17	1	600136-00-0	CABLE TIE SST1M 2.4x102 *	
18	1	1824162	PCB ANGLE STOP	
19	1	1824157	CABLE ACTUATOR 2000mm.	
20	1	305677-00-0	CABLE ACTU CONTROL L=1000 MS90 CH HEXIII	
21	1	306853-00-0	CABLE ADAPTION SEAT ELEV EL.UNIT SLS V.2	
22	1	1824158	CABLE VR2 TO PCB 4-FUNCT.	
23	1	1824159	CABLE SWITCH 4-FUNCTIONS VR2	
24	1	1824160	HOLDER SWITCH 2/4 FUNCT.	
25	1	1825387	STICKER CONTROL PANEL, 4 FUNCTIONS	
26	1	1826472	PCB MULTIPLEXER	2-4 FUNCTIONS, VR2. Including box and lid.
27	1	1822363	CABLE INHIBIT/LOW SPEED VR2 C300 C350	
28	1	1825439	CABLE ACTU TS-SEAT VR2	
29	1	1824002	CABLE SEAT TILT 30° DRIVE INHIBIT/ELEVATOR STOP	

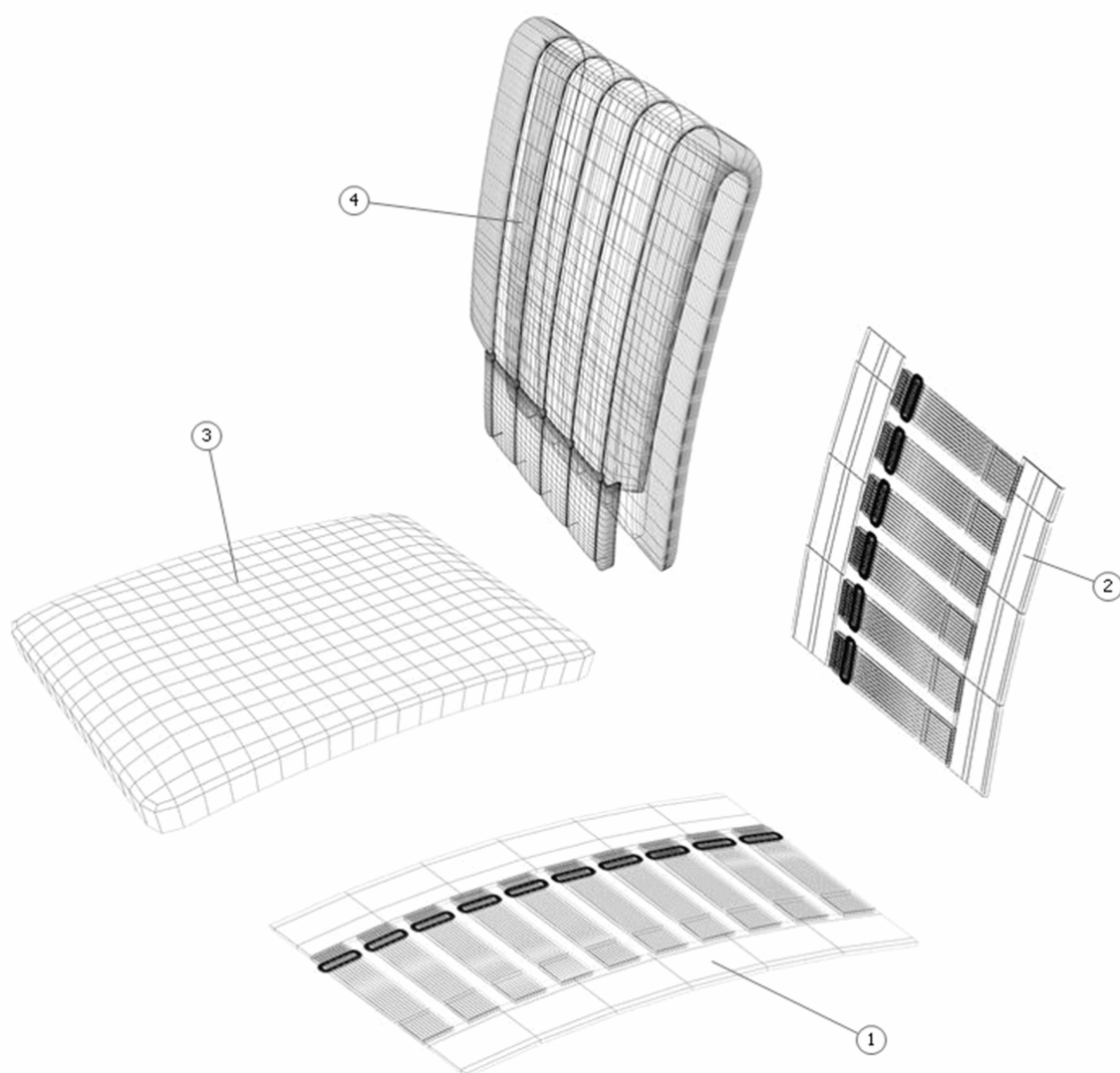

Pos	Qty	Stock Code	Description	Remark
1	1	1823718	SEAT PAN ADJ. TS-SEAT	
2	1	1823723	BACKR PAN ADJ. HIGH BACK TS-SEAT	
2	1	1823725	BACKR PAN ADJ. LOW BACK TS-SEAT	
3	1	1823716	SEAT CUSHION NARROW 430mm.	
3	1	1823717	SEAT CUSHION WIDE 480mm.	
4	1	1823714	BACKREST CUSHION NARROW 430mm.	
4	1	1823715	BACKREST CUSHION WIDE 480mm.	

When you order spare parts, the following must always be specified:

- The product model (for example, power wheelchair C300).
- The description of the part.
- always use the description used in this catalog.
- The entire part number.
- The quantity.

NB! In some cases, parts appear in the spare part catalog without a part number. This means that the part cannot be ordered from Permobil. We have, however, chosen to describe it for your information. Standard components such as screws, nuts, washers and similar parts can be purchased in an ordinary shop using the information in this catalog.

Description of Positioning list

The positioning list consist of:

A = Indicates the sparepart-group covered by the catalog.

B = Indicates part of the sparepart-group.

C = Under C there are five columns:

- 1** = Pos.no: Position number
- 2** = Pcs: Total number of given pos-no.
- 3** = Part no.: Part number
- 4** = Description: Article name
- 5** = Remark: Reserved for additional information such as reservations, restrictions etc.


Spare Part Catalog Reservdelskatalog		C300 C300		Seat Tube, Fixed Position Fast sitsrör	
①	②	③	④	⑤	
Pos no.	Pcs	Part no.	Description	Benämning	Remark/Anm.
1	1	1820446	Seat Tube	Sitsrör fast	
1	1	1820418	Seat Tube for tilt	Sitsrör fast för sitslutning	
2	1		Outer tube	Ytterrör	
3	1		Screw ISO 4762 M8x55 fzb	Skruv ISO 4762 M8x55 fzb	
4	4	1820421	Cap	Hätta	
5	1		Cable Tie Holder 10x15	Buntbandsfäste 10x15	
6	1		Screw ISO 7380 M4x6 fzb	Skruv ISO 7380 M4x6 fzb	
7	1		Nut DIN 985 M4 fzb	Mutter DIN 985 M4 fzb	
8	4		Screw ISO 4762 M6x10 fzb	Skruv ISO 4762 M6x10 fzb	
9	1	1820422	Inner tube	Innerrör	
9	1	1820419	Inner tube for tilt	Innerrör för tilt	
10	1		Screw ISO 4762 M6x10 fzb	Skruv ISO 4762 M6x10 fzb	
11	1		Bracket	Fjäder	
12	1		Stop	Stång	
13	1		Screw ISO 7380 M3x6 fzb	Skruv ISO 7380 M3x6 fzb	
14	1		SLS Bracket	SLS fäste	

Part

A part consists of one or more parts. If a part consists of several parts, a bracket in connection with the part's item bubble is used to show the parts included in the part in the exploded views. See example 1.

Quantity

The quantity specified in front of a part is the total quantity for the entire product.

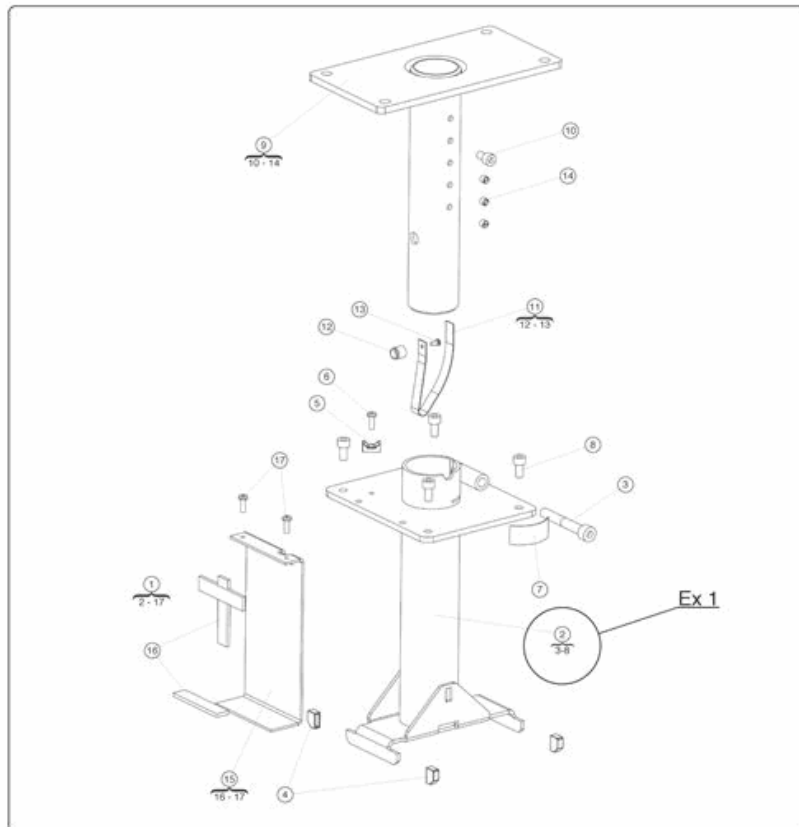**Example 1:**

The fixed seat tube, pos 2, includes pos 3 to 8, that are indicated in the bracket below (see above).

Example 2:

If a outer tube, pos 2, is ordered, the quantity is 1. In other words, there is just one outer tube on the chassis. However, if you look at the number of parts included, the quantity may vary. For example, in pos 4, the complete outer tube comprises 4 parts.

Vid beställning av reservdelar skall följande alltid anges:

- Produktmodell (ex.vis elrullstol C300).
- Artikelns benämning.
- använd samma benämning som i denna reservdelskatalog vid beställning.
- Hela artikelnumret.
- Antal.

OBS! I reservdelskatalogen förekommer i vissa fall detaljer utan angivet artikelnr. Det innebär att detaljen inte går att beställa från Permobil, men vi har ändå valt att beskriva dem för din information. Standardkomponenter så som skruv, mutter, bricka och liknande detaljer kan införskaffas i den ordinarie detaljhandeln med hjälp av informationen i denna katalog.

Beskrivning av positionslistor

Positionslistan består av:

A = Visar vilken produkt katalogen beskriver.

B = Visar vilken del av produkten som visas på aktuell sida.

C = Under C finns fem kolumner:

- 1 = Pos.no: Positionsnummer
- 2 = Pcs: Antal detaljer som ingår i aktuellt pos.nummer
- 3 = Part no.: Artikelnummer
- 4 = Benämning: Artikelns benämning (svensk och engelsk benämning)
- 5 = Anmärkning: Plats för ev. tilläggsinformation, förbehåll etc.


Spare Part Catalog Reservdelskatalog		C300 C300		Seat Tube, Fixed Position Fast sitsrör	
1	2	3	4	5	
Pos no.	Pcs	Part no.	Description	Benämning	Remark/Anm.
1	1	1820446	Seat Tube	Sitsrör fast	
1	1	1820418	Seat Tube for tilt	Sitsrör fast för sitslutning	
2	1		Outer tube	Ytterrör	
3	1		Screw ISO 4762 M8x55 fzb	Skruv ISO 4762 M8x55 fzb	
4	4	1820421	Cap	Hätta	
5	1		Cable Tie Holder 10x15	Buntbandsfäste 10x15	
6	1		Screw ISO 7380 M4x6 fzb	Skruv ISO 7380 M4x6 fzb	
7	1		Nut DIN 985 M4 fzb	Mutter DIN 985 M4 fzb	
8	4		Screw ISO 4762 M6x10 fzb	Skruv ISO 4762 M6x10 fzb	
9	1	1820422	Inner tube	Innerrör	
9	1	1820419	Inner tube for tilt	Innerrör för tilt	
10	1		Screw ISO 4762 M6x10 fzb	Skruv ISO 4762 M6x10 fzb	
11	1		Bracket	Fjäder	
12	1		Stop	Stång	
13	1		Screw ISO 7380 M3x6 fzb	Skruv ISO 7380 M3x6 fzb	
14	1		SLS Bracket	SLS fäste	

Artikel

En artikel består av en eller flera detaljer. Om en artikel består av flera detaljer är det markerat i sprängskisserna med ett klamer i anslutning till artikelns posbubbla där det står vilka detaljer som ingår i artikeln. se Ex 1.

Antal

Det angivna antalet framför en artikel är det sammanlagda antalet för hela produkten.

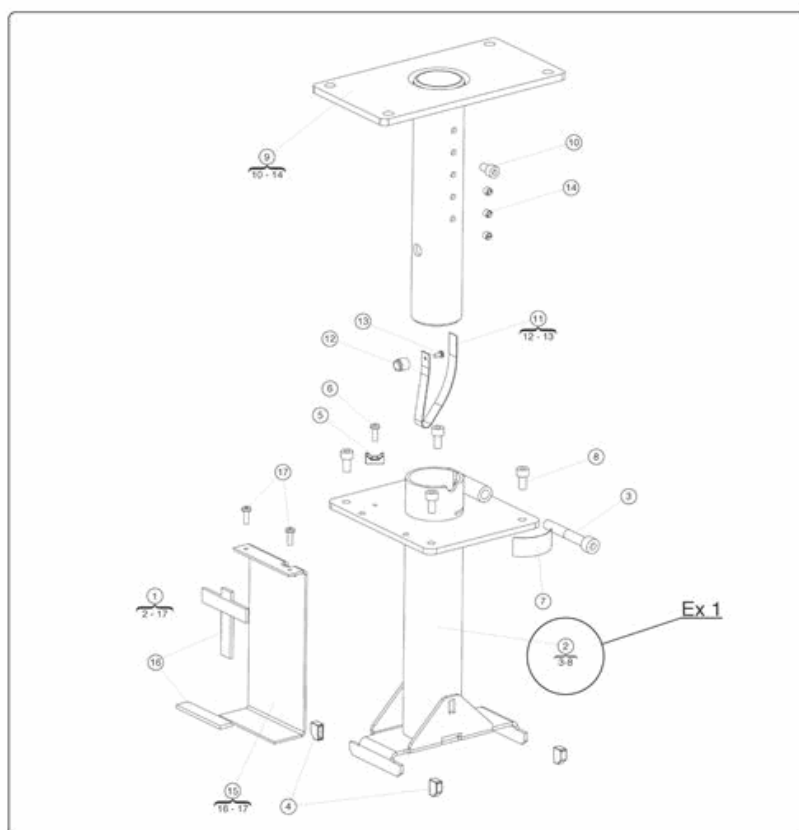**Ex 1:**

I fast sitsrör Pos. 2, ingår poserna 3till 8 som står i klamret under. (se ovan)

Ex 2:

Beställs ett Ytterrör, pos. 2, så står det att antalet är 1. Det finns med andra ord bara ett ytterrör på chassit. Men om man tittar på antalet i ingående detaljer, så kan antalet variera, i tex. pos 4 är det 4 st för att ytterröret ska vara komplett.

This image shows a full page of white paper with horizontal dashed lines, typical of primary-ruled notebook paper. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

This image shows a full page of white paper with horizontal dashed lines, typical of primary-ruled notebook paper. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

This image shows a full page of white paper with horizontal dashed lines, typical of primary-ruled notebook paper. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.



PAB no: 1256